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Descriptif du Vibby OAK

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USER GUIDE

Vibby OAK description

Your **Vibby OAK** can be worn either on the wrist or around the neck as a pendant. The **Vibby OAK** should be worn at all times when you are in or around your home.

Technical specifications

Weight: 35g	Radio frequency : 869.2125 MHz / 869.2375 MHz
Water resistant	Radio range : 200m outdoors
Cleaning: soap and water only	Operating temperature range: +5° C - +40° C
Watchband: hypoallergenic rubber	Battery life*: CR2477 / LiMnO ² - Up to 2 years

*The battery replacement will be done by your installer

Vibby OAK
The installer will configure your Vibby OAK into either wristband or pendant mode depending on your needs. You should not change the Vibby OAK's mode without contacting your service provider.

Raising an alarm

Manual alarm: ▶ Triggered by pressing the help button on your **Vibby OAK**.

Automatic alarm: ▶ When the **Vibby OAK** detects a fall it automatically sends an alarm to the Home unit.

Vibrator: Before automatic alarm transmission to the Home unit, the **Vibby OAK** will vibrate and the red LED will flash for 20 seconds intermittently. If you wish to cancel the alarm call before the transmission, place your hand over the **Vibby OAK** until the vibrations stop.

Please note :

Some falls (i.e. soft falls, controlled descent against a wall or into a chair) may not be detected by the **Vibby OAK**. Fall detection technologies that underpin the **Vibby OAK** solution do not analyse / interpret all situations. Should you have a fall and you do not feel the vibrations from the **Vibby OAK** just press the help button on the **Vibby OAK** to manually raise an alarm call.

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Certificate of compliance

We hereby certify that the **Vibby OAK** by the Vitalbase division of Telecom Design complies with the main requirements and associated regulations presented in directives 1999/5/EC (R&TTE), 2011/65/EU (RoHS-2) and EC 1907/2006: NO_SVHC REACH. A copy of the Declaration of compliance is available for consultation at <http://docs.vitalbase.fr>

WEEE

Eliminating household Waste Electrical and Electronic Equipment in the European Union. The presence of this symbol on a product or packaging indicates that you should not dispose of this product with other household waste. On the contrary, it is your duty to dispose of waste equipment by depositing it at a dedicated collection point for recycling Waste Electrical and Electronic Equipment. For further information on these collection points, please contact your local town hall, waste disposal service or dealer. When batteries are collected at the same time as Waste Electrical and Electronic Equipment in accordance with directive 2002/96/CE, they must first be removed from appliances.

vitalbase

MATP0498 V4

TELECOM DESIGN Health care Division

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