
USER GUIDE

En

!!! Please refer to our website: www.geemarc.com for an up to date user guide, as there may be important updates and changes you need to be aware of !!!

DESCRIPTION OF THE PRODUCT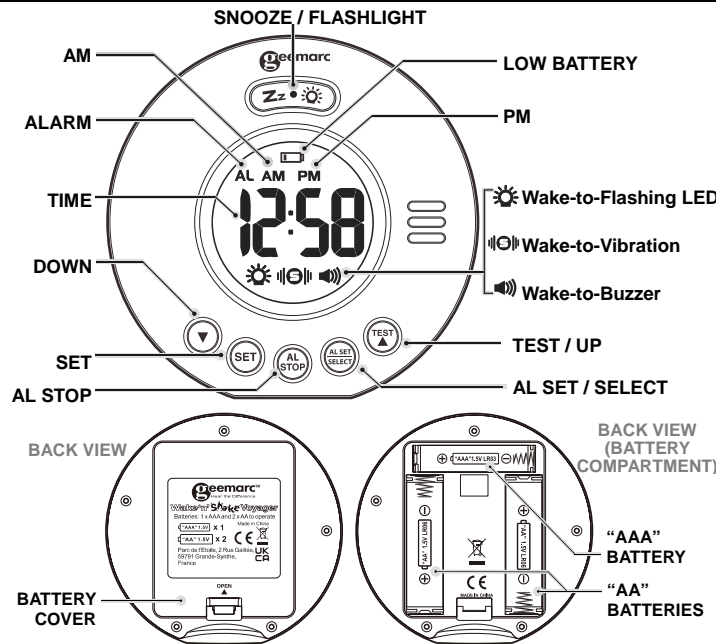

BATTERY INSTALLATION

Your clock requires 1 x AAA and 2 x AA alkaline batteries (included) to operate.

- Place your unit face down on a flat surface.
- Press the latch and remove the battery cover at the back of your unit.
- Insert 1 x AAA and 2 x AA new alkaline batteries into the battery compartment by observing the correct "+" and "-" polarity.
- Replace the battery cover.

TO SET TIME AND THE SNOOZE DURATION

- Press and hold **SET**, the hour digits "00" flash. Press **TEST/▲** or **▼** to set the time in hours (hold **TEST/▲** or **▼** to advance rapidly).
- Press **SET** again, the minute digits "00" flash. Press **TEST/▲** or **▼** to set the time in minutes. (hold **TEST/▲** or **▼** to advance rapidly).
- Press **SET** again, the display shows "24Hr" and flashes. Press **TEST/▲** or **▼** to select "12Hr" or "24Hr" time format. There has AM icon to indicate the morning time and PM icon to indicate the afternoon time.
- Press **SET** again, the display shows "05" and flashes. Press **TEST/▲** or **▼** to select your snooze time from 5 to 60 minutes.
- Press **SET** again or if no key pressed for approx. 10 seconds to exit the setting mode.

TO SET THE ALARM (default at 6:00 AM)

- Press and hold **AL SET / SELECT**, the AL appear and the alarm's hour digits' flash. Press **TEST/▲** or **▼** to set the alarm in hours. (hold **TEST/▲** or **▼** to advance rapidly).
- Press **AL SET / SELECT** once again, the alarm's minute digits' flash. Press **TEST/▲** or **▼** to set the alarm in minutes. (hold **TEST/▲** or **▼** to advance rapidly).
- Press **AL SET / SELECT** again, **||⊖||** and "HI" flash (vibration at HI level), press **TEST/▲** or **▼** to select "LO" (vibration at Low level) if need.
- Press **AL SET / SELECT** again, **||⊖||** and "HI" flash (Sound at HIGH level), press **TEST/▲** or **▼** to select "LO" (Sound at Low level) if need.
- Press **AL SET/SELECT** once again or no button around 10 seconds to exit the alarm setting mode. Admist alarm setting, press **SET** once to exit and return to nomral time

display mode.

TO SELECT THE ALARM MODE

At normal time display mode, press **AL SET / SELECT** one at a time to activate the alarm(s), the corresponding alarm indicator(s) on the right side of the time display will be on:

only sound alarm is activated	only shaker alarm is activated	only strobe light alarm is activated	both sound and vibration alarm are activated	both strobe light and sound alarm are activated

Press **AL SET / SELECT** once more to turn off all the alarms, all alarm indicators disappear.

Remark: frequent use of the vibration and flash light will quickly shorten the battery life.

TO STOP AND RESET THE ALARM TO COME ON THE NEXT DAY

When alarm is sounding, the corresponding alarm indicator flashes. Press **AL STOP** once to stop the alarm and reset it to come on the following day. After that, the corresponding alarm indicator stays on the display.

TO USE SNOOZE

When alarm is sounding, presses **Zz • ☀** once, the alarm will be stopped temporarily and the corresponding alarm indicator keeps flashing. Alarm will come on again after the set snooze duration is over.

TO USE BACKLIGHT

Press **Zz • ☀** once to active backlight 5 seconds.

TO USE FLASHLIGHT

Press and hold **Zz • ☀** to active flashlight. Press and hold **Zz • ☀** to de-active the flashlight.

TEST MODE

Press and hold **TEST/▲** Button, strobe light flashing, shaker vibrating and buzzer sounding for 5 seconds and then stops.

LOW BATTERY ALERT

When the battery level becomes low, a low battery icon will appear on the upper middle of the LCD. Repalce batteries with new alkaline batteries.

Remark: frequent use of the vibration alarm will quickly shorten the battery life.

TROUBLE SHOOTING

If your clock displays irrelevant time or does not function properly, which may be caused by electro-static discharge or other interference, open the battery cover, take out the batteries and reinstall them by observing the correct +/- polarity. Your clock will be reset to default settings and you need to set it again.

Low battery level will decrease the vibration and the alarm sound. Once this happens, replace batteries with new ones.

CARE OF YOUR PRODUCT

- Place your clock on a stable surface, away from sources of direct sunlight or excessive heat or moisture.
- Protect your furniture when placing your units on a natural wood and lacquered finish by using a cloth or protective material between it and the furniture.
- Clean your unit with a soft cloth moistened only with mild soap and water. Stronger agents such as Benzene, thinner or similar materials can damage the surface of the unit. Make sure the unit is unplugged before cleaning.
- If the unit is not to be used for a prolonged period, such as a month or longer, remove the batteries to prevent possible corrosion. Should the battery compartment become corroded or dirty, clean the compartment thoroughly and replace the batteries.

SPECIFICATIONS

Alarm duration	: 1 minute	
Snooze duration Adjustable	: 5 to 60 minutes	
Default settings	Time format	24 hour
	Time	0:00
	Alarm	6:00
	Shaker alarm level	Hi
	Snooze duration	05 (5 minutes)
Audible alarm	: up to 75 dB at 1m	

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at www.geemarc.com. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

DECLARATIONS

UKCA Directives: Geemarc Telecom SA hereby declares that this Wake 'n' shake Voyager is in compliance with the UKCA Directive requirements.

The UKCA declaration of conformity may be consulted at www.geemarc.com

Recycling Directives: The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.

- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website at www.geemarc.com

E-mail: help@geemarc.com
telephone 01707 387602

lines are open 09h00 to 16h00 Mon to Fri
2 rue Gallée, Parc de l'Etoile, 59791 Grande-Synthe Cedex, France

This product is made in China To Geemarc Telecom

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