geemarc™ AMPLICALL™ 101

Hear the Difference

Wireless doorbell
Sonnette sans fil
Drahtlose Türklingel

English P1

Français P12

Deutsch P23


!!! Please refer to our website:
www.geemarc.com for an up to date user guide,
as there may be important updates and changes
you need to be aware of !!!

PACKAGE CONTENTS

The package contains the following items:

- 1 user's manual
- 1 AMPLICALL™101 (AC101)
- 2 screws and 2 fixing holes
- 1 wall bracket and 1 lanyard bracket.


DESCRIPTION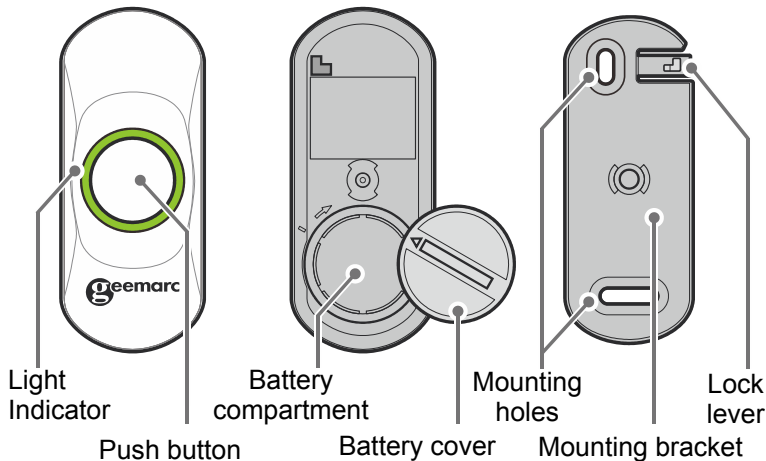

INSTALLATION

Battery installation or replacement in the remote

AMPLICALL™ 101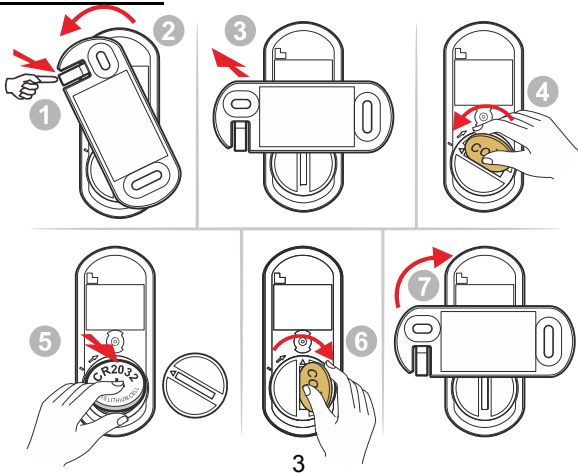


1. Use your finger to unlock the lock lever (fig ①).
2. Rotate the mounting bracket to 90 degrees (fig ②).
3. Remove the mounting bracket from the unit (fig ③).
4. Rotate the battery cover using a coin (fig ④).
5. Insert a CR2032 battery into the battery compartment with the polarity + on top. (fig ⑤).
6. Close the battery cover using a coin. (fig ⑥).
7. Replace the mounting bracket to 90 degrees. (fig ⑦).
8. Rotate the mounting bracket until the part locks.

Note: Low Battery Signal

When the AC101 is running low on battery, it will send a signal to the AMPLICALL™100 or AMPLICALL™150 which will make the corresponding zone icon light up. To replace the battery, you will need to follow the 8 steps procedure described above.

CODE CONFLICT

If the bell press coding is the same as another transmitter you must change the code. When you have changed the code must re-pair the bell press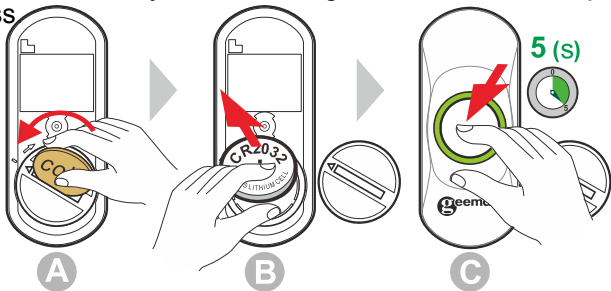

(A) Open the battery cover.


(B) Take out the battery.

(C) Press and hold the test button, insert the battery whilst holding the button for 5 seconds. The LED will flash 5 times when a new code is generated.

Pairing with AMPLICALL™100/AMPLICALL™150


Enter pairing mode on the AC100 or AC150:

1. Press & hold the **LEARNING** button (●) for 3 seconds. The LED in zone icon #1 will flash.
2. Press the **LEARNING** button (●) again to move to the zone icon you wish to pair the AC101 with. Press the **melody selection** (🎵) button to scroll through the melodies. Once you hear a melody you like, do not press any button during the melody.
3. Press once the doorbell button of the AC101 to pair.
4. If successful, the AMPLICALL100 or 150 will generate two short tones.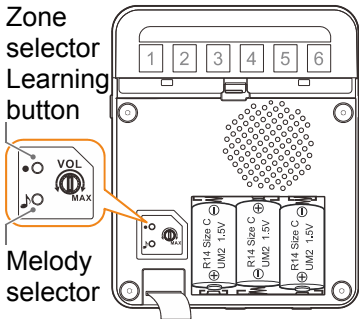


WALL MOUNTED INSTALLATION

Mounting with screws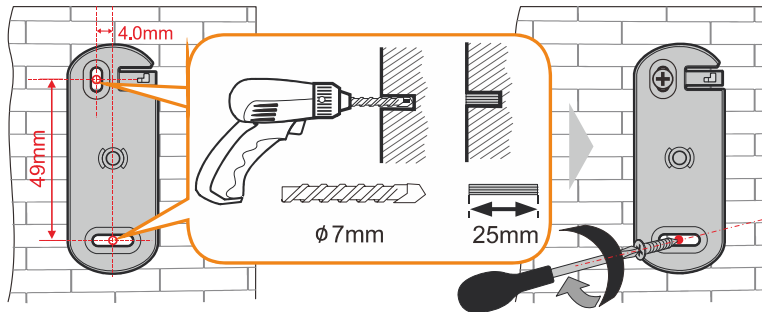


Requires an electric drill to fix the AC101 on the wall using the provided ram plug and screws.

Mounting with Double-sided adhesive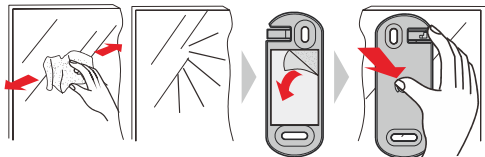


You can install your AMPLICALL™101 onto a wall.
Please, follow the recommendations below.

Attach the Doorbell to the mounting bracket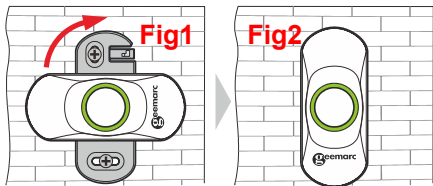

1. Replace the mounting bracket to 90 degrees. (fig 1).
2. Rotate the mounting bracket until the part locks. (fig 2).

Put the bell press to the lanyard bracket

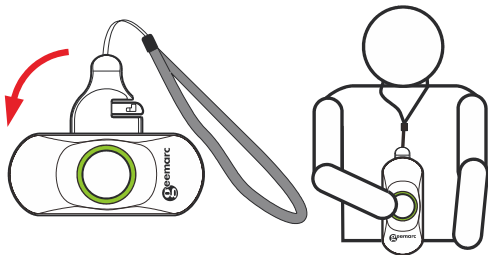1. Place the lanyard bracket to 90 degrees.
2. Rotate the mounting bracket until the part locks.

GUARANTEE

From the moment your Geemarc AMPLICALL™101 is purchased, Geemarc guarantee is for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit

our website at **www.geemarc.com**.

The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

DECLARATIONS:

UKCA Directives

Geemarc Telecom SA hereby declares that this AMPLICALL™101 is in compliance with the essential requirements and other relevant provisions of the Radio Equipment Directive 2014/53/UE.

*The UKCA declaration of conformity may be consulted at **www.geemarc.com***

RF power : < 250mW

Recycling Directives

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website at www.geemarc.com

E-mail : help@geemarc.com telephone **01707 387602**

lines are open 09h00 to 16h00 Mon to Fri

2 Rue Galilée, Parc de l'Etoile, 59791 Grande-Synthe Cedex, France

This product is made in China To Geemarc Telecom

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