# AMPLICALL20

Telephone ringer amplifier With flasher and door bell

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!!!Please refer to our website: [www.geemarc.com](http://www.geemarc.com) for an up to date user guide, as there may be important updates and changes you need to be aware of!!!

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
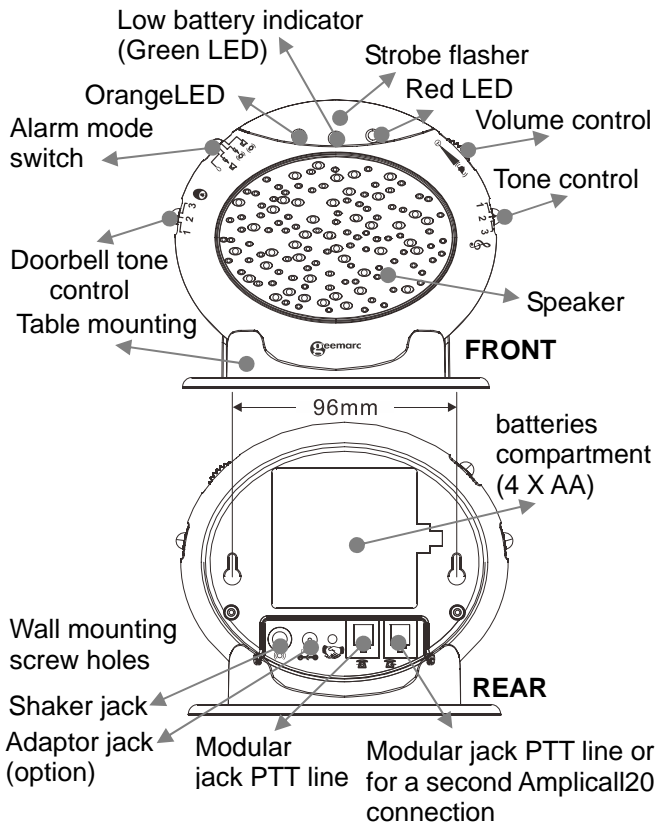
# INSTALLATION

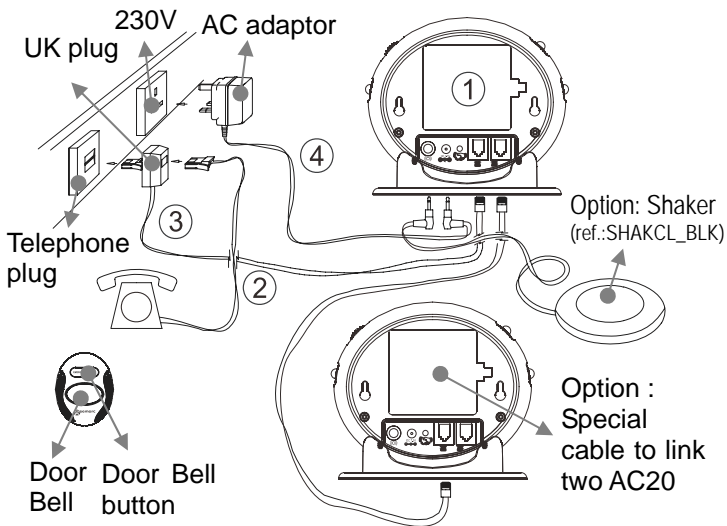
## Installation (See drawing)

- 1 - Insert 4 AA batteries into the battery compartment
- 2 - Connect the UK line cord into your telephone and into the connector fitted from the Amplicall 20 into the PTT socket.
- 3 - Connect the UK line cord from the Amplicall 20 and the PTT wall socket.
- 4 - Plug the power cable into the power adapter jack found at the rear of the Amplicall 20. Plug the adaptor into the mains supply at the wall (optional extra)
- 5 - Insert the 23A 12V battery into the door bell.

# INSTALLATION
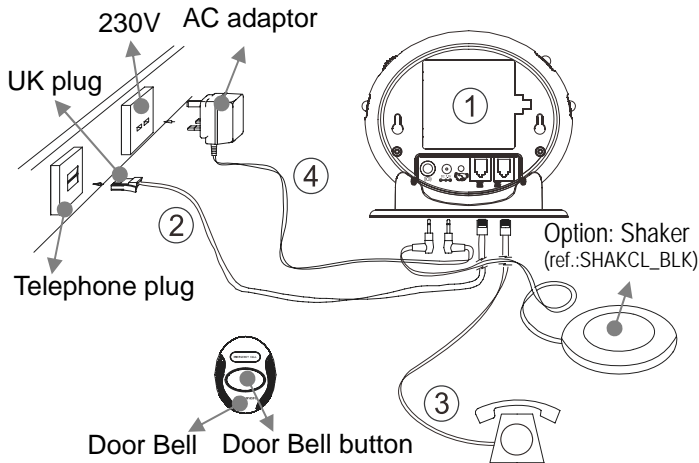
## Second method:

- 1) Insert 4 AA batteries into the battery compartment
- 2) Connect one end of the telephone line cord to the Amplicall 20 and the other end to the wall socket.
- 3) Connect the RJ11 cable between the Amplicall 20 and the telephone.
- 4) Plug the power cable into the power adapter jack found at the rear of the Amplicall 20. Plug the adaptor into the mains supply at the wall (optional extra).
- 5) Insert the 23A 12V battery into the door bell.

# INSTALLATION

**Electrical connection :** The apparatus is designed to operate from a 230V 50Hz supply only. (Classified as «hazardous voltage» according to EN60950 standard). The apparatus does not incorporate an integral power on/off switch. To disconnect the power, either switch off supply at the mains power socket or unplug the AC adaptor. When installing the apparatus, ensure that the mains power socket is readily accessible.

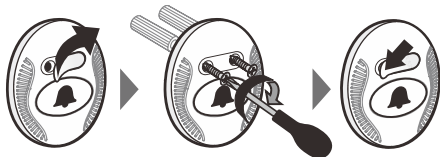
**Telephone connection :** Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN60950 standard.

# INSTALLATION

## Setting up Door bell

---

1. The door bell can be used by people who need assistance.
2. The door bell can be fixed on a wall or a door surround. Drill two holes horizontally, 15mm apart and place the supplied anchors into the two holes. Hang the door bell onto the two anchors and then insert and tighten the screws. Stick the EMERGENCY CALL sticker over the two holes.

3. The door bell can be stuck onto a door or window using the supplied strong adhesive tape. Remove the protective paper from the adhesive tape on the back of the door bell and stick to the required door or window.

## Wall Mounting

---

1. Drill two holes horizontally, 96mm apart, and leave the screws protruding from the wall by 5mm.
2. Hang the base on to the two screws and pull the base down to lock it on

# INSTALLATION

## Table Mounting

---

If you want to mount the unit on a table, simply place it in the base cradle and place the base cradle on a table.

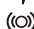
## Setting the Alarm Mode

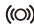
---

Choose which alarm mode you want by moving the alarm mode switch to one of the following positions:

0 :no alarm

 :both strobe flasher and sound alarm activated  

 :both strobe flasher and shaker (if connected)  
 activated

 :both sound alarm and shaker (if connected)  
 activated

## Volume Control Slide Switch

---

Use the slider control to adjust the volume of the door bell ring, phone ring or power OFF

## Tone Control Switch (3 Positions)

---

Set the tone of the alarm sound.

## Door Bell Tone Control

---

The three position switch allows you to set a different tone when the door bell rings.

# INSTALLATION

## Change The Battery of Indoor unit (option)

The unit requires 4xAA batteries

1. Open the battery compartment door.
2. Insert 4 x AA batteries into the battery compartment.
3. Snap the battery compartment door back into place.

**Note:** when the red LED indicator flashes, it is time to replace the batteries.

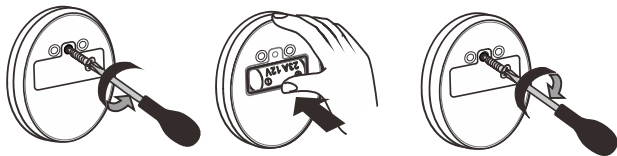
Whilst replacing the batteries, disconnect all telephone line cords from the modular wall jacks.

## Second Amplicall 20

Another Amplicall 20 can be added to the system (call **01707 387602**)

## Install and Change the Battery of the Door bell


Install the 23A 12V battery into the door bell

To replace the battery, unscrew the door bell from the wall, and unscrew the one screw located on the back to gain access to the battery compartment. Replace the battery then replace the battery compartment flap and screw and remount.

# INSTALLATION

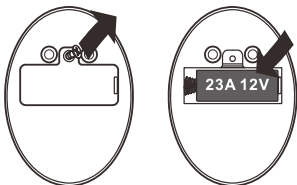**Note:** Do not use a rechargeable battery. Do not put the used battery into the household rubbish, pay attention to regulations regarding battery disposal.

## The Pairing Procedure

---

1. Press and hold the Pairing/Test button until the unit makes a beep sound, then release the Pairing/Test button. The 1<sup>st</sup> LED lights to indicate that the pairing mode has been entered.
2. Press and hold the Pairing/Test button again until the 1<sup>st</sup> LED flashes, then press and hold the remote button at the same time until the strobe flasher flashes. The unit makes a beep sound to confirm pairing has been successful.
3. If the user wants to pair a 2<sup>nd</sup> and 3<sup>rd</sup> remote, when the 1<sup>st</sup> LED lights, press the Pairing/Test button once, the 1<sup>st</sup> LED goes off and then the 2<sup>nd</sup> LED lights. The user is then able to pair the 2<sup>nd</sup> remote. Repeat the same procedure for the 3<sup>rd</sup> remote.
4. Repeat the above, all 3 LEDs flashing together for the 4<sup>th</sup> remote pairing.

# INSTALLATION

## **Un- Register The Pairing:**

---

1. Press and hold the Pairing/Test button for 5 seconds to enter un-register mode, 3 LEDs flash together.
2. If user wants to un-register the 1<sup>st</sup> door bell, press the Pairing/Test button once, the 1<sup>st</sup> LED will be flashing (the other 2 LED light), then press and hold the Pairing/Test button until the 1<sup>st</sup> LED goes off, beeps to confirm. The 1<sup>st</sup> door bell is now un-registered.
3. If user wants to keep the 1<sup>st</sup> door bell but un-register the 2<sup>nd</sup> or 3<sup>rd</sup> door bell, when the 1<sup>st</sup> LED flashes, press the Pairing/Test button again, the 2<sup>nd</sup> LED flashes (other 2 LED light) to select which door bell to be un-registered.
4. Repeat the above, all 3 LEDs flashing together for the 4<sup>th</sup> door bell unregister.

To check whether the unit works normally, you can press the Door bell button, the unit would sound alarm and flash.

## **Operation**

Once the indoor unit and the door bell have been placed in the required locations and all required batteries have been inserted, you are ready to use the device.

To use the device as a door bell, the person who needs assistance must press the Door Bell button. The indoor unit will sound an alarm and flash (depending on the Alarm mode setting).

To use the device as a telephone ringer amplifier whilst receiving a new call, the indoor unit will sound an alarm and flash (depending on the Alarm mode setting)

# TROUBLESHOOTING

## Trouble shooting

Symptom	Solution
No incoming call yet the Amplicall20 rings automatically or rings continuously. The shaker vibrates automatically or continuously. The strobe LED flashes automatically or continuously.	<ol style="list-style-type: none"><li>1) Check the batteries are inserted correctly.</li><li>2) Check the power adaptor is connected correctly.</li><li>3) Turn off the power for 10 minutes and then turn it back on.</li></ol>
The red low battery LED flashes automatically.	<ol style="list-style-type: none"><li>1) Replace the batteries.</li><li>2) Turn off the power for 10 minutes and then turn it back on.</li></ol>
When there is an incoming call but no ringing can be heard.	<ol style="list-style-type: none"><li>1) Check it is connected to the power adaptor correctly.</li><li>2) Check the batteries are inserted correctly.</li><li>3) Check the volume switch is on.</li><li>4) Check it is connected to the line cord correctly.</li><li>5) Check the alarm mode selection switch position is correct</li></ol>

# TROUBLESHOOTING

<p>When there is an incoming call but the shaker doesn't vibrate.</p>	<ol style="list-style-type: none"><li>1) Check it is connected to the shaker correctly.</li><li>2) Check the shaker socket to see if it has been damaged.</li><li>3) Check the alarm mode selection switch position is correct.</li></ol>
<p>The Amplicall20 does not ring when the doorbell is pressed</p>	<ol style="list-style-type: none"><li>1) Check that the red light on the doorbell flashes when it is pressed. If it does not, change the battery.</li><li>2) Register the doorbell to the AmpliCALL20 again</li><li>3) Check the alarm mode selection switch position is correct i.e. not on '0'</li></ol>

# GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at **[www.geemarc.com](http://www.geemarc.com)**. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

**IMPORTANT:** YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM

**Please note :** The guarantee applies to the United Kingdom only.

**Declaration :** Hereby Geemarc Telecom SA declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/53/UE and in particular article 3 section 1a, 1b and section 2.

*The declaration of conformity may be consulted at **[www.geemarc.com](http://www.geemarc.com)***

# GUARANTEE

**Recycling directives** : The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way. When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

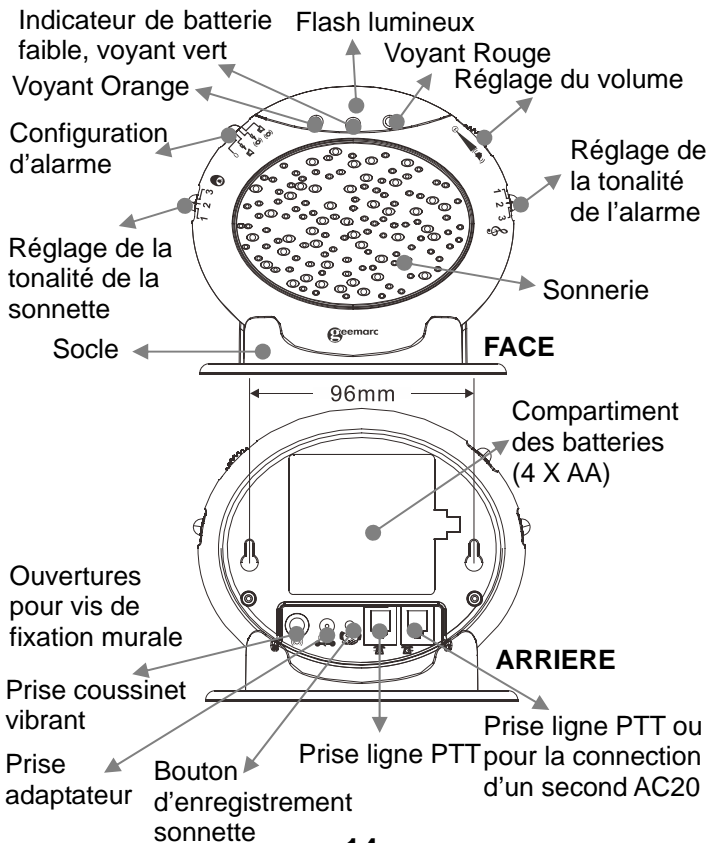
- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it. Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website  
at [www.geemarc.com](http://www.geemarc.com)  
Telephone **01707 387602**  
Or fax **01707 832529**

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