GEEMARC

CLEAR SOUND™

HEAR THE DIFFERENCE

**CL64**

**Amplified Classic Phone  
Téléphone Vintage Amplifié  
Lautverstärktes Retrotelefon**

**English 1~12**

**Français 13~24**

**Deutsch 25~36**

**geemarc™**  
Telecom SA

User manual to be read carefully before use!

**CAUTION:** this telephone is specifically designed for people with hearing difficulties and is provided with a handset earpiece volume control for individual requirements. Due care must be taken by all users that the handset earpiece volume control is set to the lowest level acceptable by each respective user. Care should therefore be exercised to ensure that any naive users are familiar with this requirement.

# CONTENTS


<b>CONTENTS</b>	<b>2</b>
<b>DESCRIPTION</b>	<b>3</b>
General Description	3
<b>SETTING UP</b>	<b>5</b>
1 - Connecting the telephone	5
2 - Ringer volume setting	5
3 - Auto Dial Switch	6
4 – Auto On Hook	6
<b>VOLUME AND TONE ADJUSTMENT</b>	<b>7</b>
1 - Receiving volume and tone adjustment	7
2 - Sending volume adjustment	8
<b>USING THE PHONE</b>	<b>9</b>
1 - Answer a call	9
2 - Making a call	9
3 - Storing telephone numbers	9
4 - Dialing stored numbers	10
<b>GUARANTEE</b>	<b>11</b>
<b>GENERAL INFORMATION</b>	<b>12</b>

# DESCRIPTION

## General Description

---


Battery low/  
No BatteryLED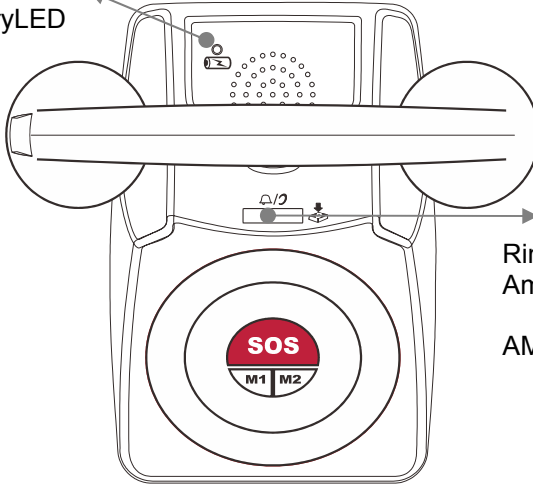

Ringer LED &  
Amplify On LED  
&  
AMPLIFY On/Off button

4XAA Size  
Battery  
Compartment

Battery door  
locking tab

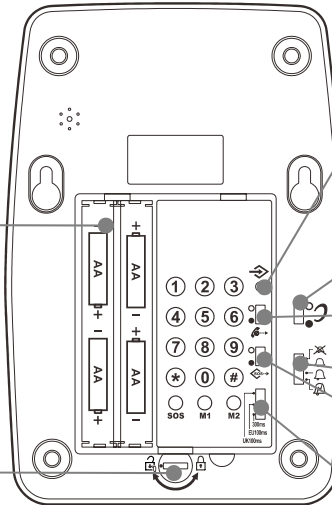Store Button

Amplify ON/OFF  
slide switch



Auto turn off Switch

Ringer Loud/High/Low/  
OFF Switch

Auto Dial SOS number  
ON/OFF Switch

Ringer UK/EURO Select  
Switch

# DESCRIPTION
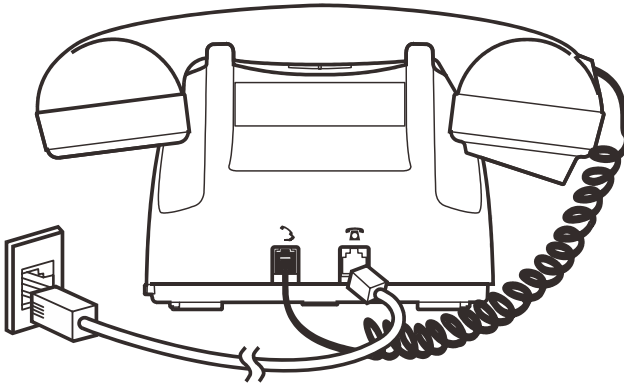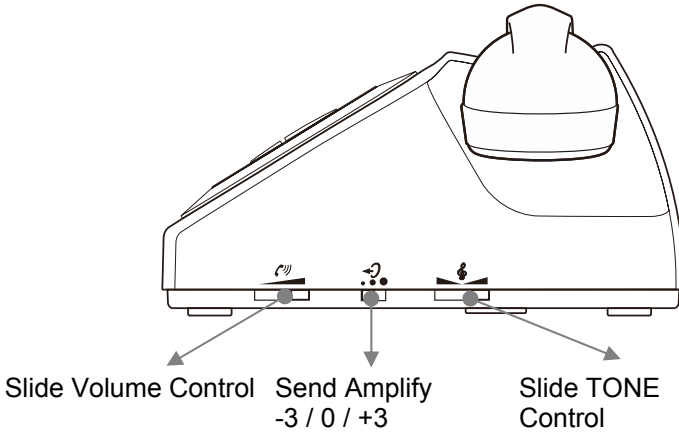
# SETTING UP

## 1 - Connecting the telephone

---

- Connect the handset cord.
- Open the battery door by turning around the locking tab. Insert 4 alkaline batteries (type AA 1.5V, not supplied) in the compartment (\*). Batteries are required if you wish to increase the ringer level up by an additional 10 dB & also use the “Mechanical Bell” ringer feature. Replace the battery door and turn the locking tab to the locked position.
- Connect the line cord plug into the line socket on the rear of the unit (\*).
- Connect the other end of the line cord into the wall PTT socket (\*).
- If inserting 4 x AA sized batteries, check that the battery low LED is extinguished. If not, replace with different batteries.

## 2 - Ringer volume setting

---

The phone is equipped with “Mechanical Bell” ringers which can be selected between UK (tring..tring) & European (triiiiiiiiiiiiing) style bells, via the switch located in the battery compartment.

The mechanical bell **sound** is functional only when the batteries are inserted and are not run down.

When there are no batteries or the batteries are “low” the Battery low indicator flashes and the ringer sound is the normal two tone electronic ring.

A switch located on the right side allows the ringer to be switched to LOUD Hi, Low or Off:

 : Off

 : Low

# SETTING UP

 : Hi

 : LOUD

**Note:** *The ring level will be increased of 10 dB if you insert batteries.*

## 3 – Auto Dial Switch

---

When this switch is in ON position (●), the phone will automatically dial out the number stored in the SOS memory if no other dial button is pressed for 4 seconds. Set this switch as per your requirement.

## 4 – Auto On Hook

---

When this switch is in the On position (●), the phone will automatically go On Hook (disconnect the telephone line) when the person at the other end disconnects the line, even if the handset is not replaced on the cradle.

In case the handset is removed from the cradle and there is a phone call, the phone will still ring as usual. To answer the call, pick up the handset, press and release the hook button once.

# VOLUME AND TONE ADJUSTMENT

## 1 - Receiving volume and tone adjustment

---

You can increase the volume and adjust the receiving tone according to your own hearing requirements.



When the switch  located under the phone is **ON** (●), the extra-receiving and tone functions are automatically activated each time you use the phone.

When this switch is **OFF** (○), you will need to manually activate the function by pressing the button .

Pressing the button  (AMPLIFY ON/OFF switch) allows extra receiving and tone functions to be activated during the conversation.

When the function is activated, the red light of the extra receiving and tone function located below the ringer LED lens lights up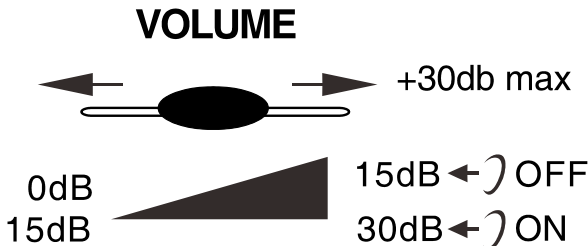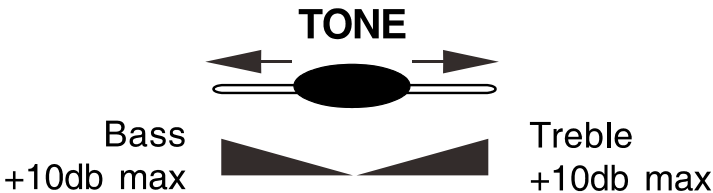

Adjust the **TONE** and **VOLUME** buttons to fit the levels to your own hearing requirements.

# VOLUME AND TONE ADJUSTMENT

**Note:** When the tone (treble), earpiece volume and sending levels are set to the maximum, whistling may occur. Then the sending level settings should be decreased with the switch .


When the switch  located in the bottom of the phone base is **ON** () , the extra-receiving and tone functions are automatically activated each time you use the phone. When this switch is **OFF** () , you will need to activate the function by pressing the button .

Tone and volume (15dB to 30dB) adjustment is only activated when the button  is set to **ON** () . When this button is set to **OFF** () only the volume can be adjusted from 0dB to 15dB.

## 2 - Sending volume adjustment

---

If your voice is quiet or called parties have difficulty hearing you, you can increase the sending level volume with the switch located on the right side. You can also decrease the sending volume if required.

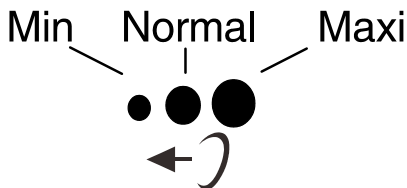# USING THE PHONE

*If you are using this telephone with a hearing aid, set your hearing aid to the T setting.*

## **1 - Answer a call**

---

- When an incoming call is received the phone will ring and the ringer LED will flash, to answer the call lift the handset and speak.
- On completion of the call, carefully replace the handset in the cradle to release the line. See also section on Auto/Manual switch

## **2 - Making a call**

---

- Lift the handset.
- Await the dial tone and immediately press the SOS, M1 or M2 buttons once
- On completion of the call, carefully replace the handset in the cradle.  
See also section on Auto Dial On/Off switch

## **3 - Storing telephone numbers**

---

Your telephone has a SOS memory button with 112 stored as default and also two other memory buttons, M1 & M2

This SOS number can be changed to a number of your choice which can be up to 18 digits. To change the SOS memory:

# USING THE PHONE

- Remove the handset from the cradle
- Press the hidden STORE button located inside the battery compartment underneath the phone
- Use the dial keypad to enter the new number
- Press SOS memory button and then replace the battery cover
- Replace the handset on the cradle

You can also store numbers of your choice (eg. Doctor, Ambulance etc) in the M1 & M2 locations using the same procedure, only press M1 or M2 button instead of SOS.

## 4 - Dialing stored numbers

---

- Lift the handset and await the dial tone.
- Press **SOS**, **M1** or **M2** Button, the stored number will be dialled automatically.

Note: If Auto Dial On/Off switch is set to ON position (●), if a telephone number key is not pressed within 4 seconds of lifting the handset, the number stored in SOS memory will automatically dial out.

# GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at [www.geemarc.com](http://www.geemarc.com). The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

**IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.**

**Please note: The guarantee applies to  
the United Kingdom only**

# GENERAL INFORMATION

## Recycling directives:

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products in end of life are recycled in the best way.

When this product is out of order, please do not put it in your domestic waste bin.

Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in a appropriate WEEE skip.
- Or hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

**Telephone connection:** Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN60950 standard.

The telephone does not operate if the line current is lower than 18mA.

**Declaration:** Hereby Geemarc Telecom SA declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC and in particular article 3 section 1a, 1b and section 2.

*The declaration of conformity may be consulted at [www.geemarc.com](http://www.geemarc.com).*

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