geemarc™ LoopHEAR™ 20
Hear the Difference

**Amplified Personal Listener
Amplificateur portatif
Mobiler Verstärker**

English 1

Français 12

Deutsch 23

INTRODUCTION

LoopHEAR™20 is a Personal Sound Amplifier which can be used by people with hearing impairment who do not have “T” coil equipped hearing aids. It amplifies speech together with sounds from a variety of sources like TV, radio and telephone.


The LoopHEAR™20 can be used as an alternative to Hearing Aids.

WARNING: This product can greatly amplify the sound. Excessive sound pressure levels can cause hearing damage so it is recommended to initially set the volume to a low level and gradually increase it to a comfortable level.

PRODUCT CONTENTS

(1) Charger

Front View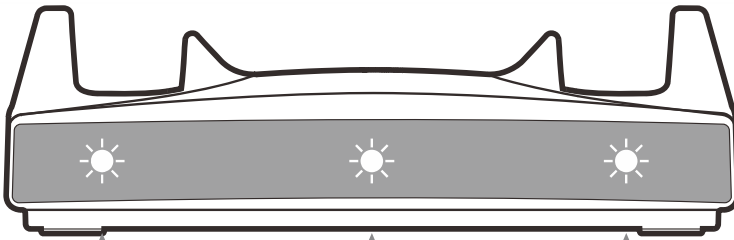


Power indicator (Blue)

Left Cradle Charging (Red)
/Full-Charged Indicator (Green)

Right Cradle Charging (Red)
/Full-Charged Indicator (Green)


Back View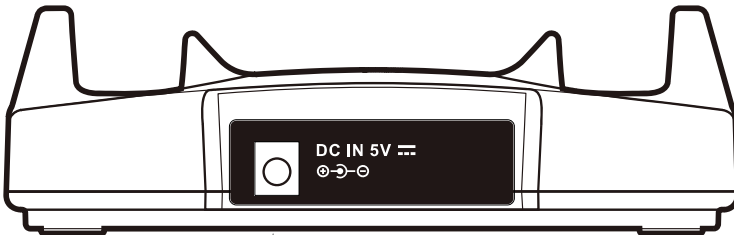

DC 5V Power Jack

NOTE: The charger can charge one or two headsets.

(2) Headset

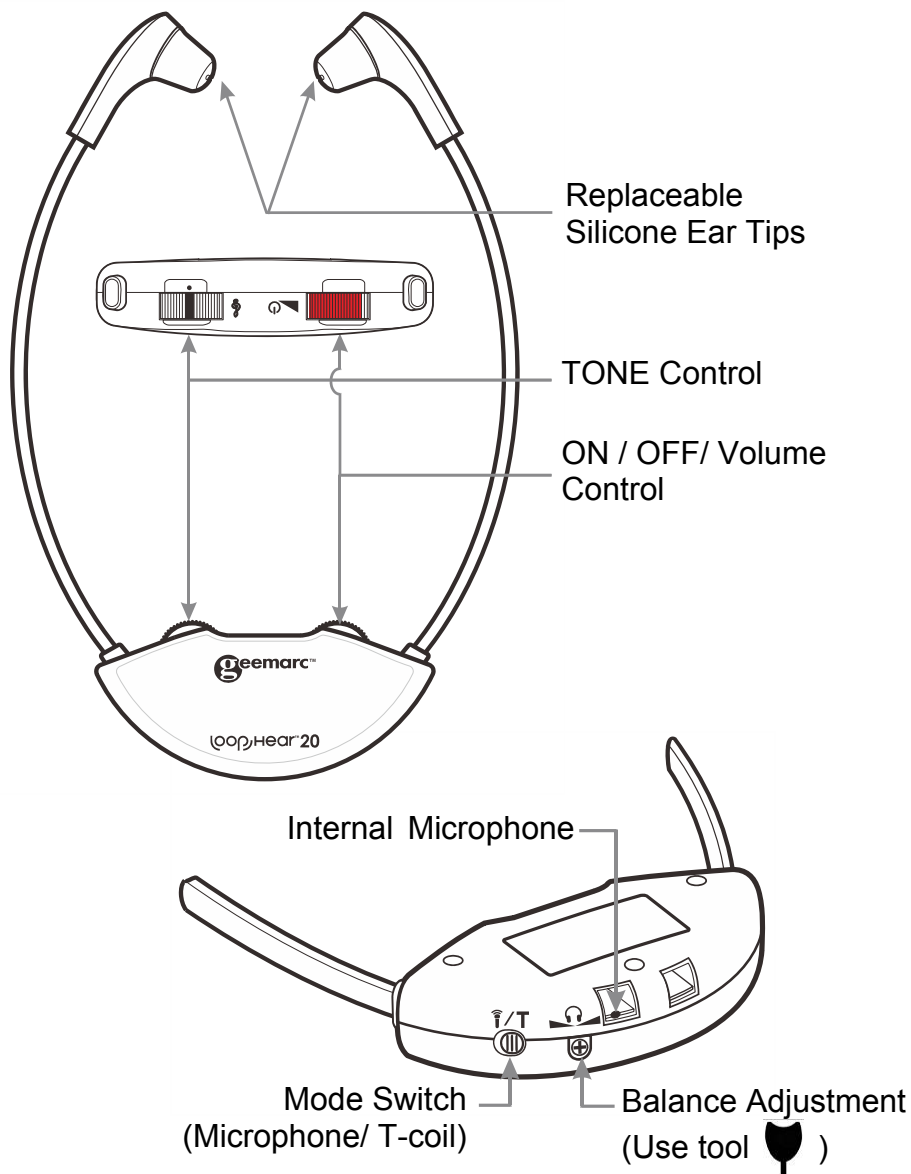NOTE: Depending on the model purchased there may be either one or two headsets in the box.

(3) 5V Power Supply (AC – DC Adapter, 100 - 240V)

(4) Spare ear tips.

(5) Lithium Polymer Rechargeable battery (Sealed inside the headset) 3.7V/200mAH

NOTE:

- 1. This type of battery should last for many years and has over 300 recharging cycles.*
- 2. We recommend that the battery be charged for about **2 hours** before the system is used for the first time.*

BATTERY WARNINGS

- Do not attempt to change or remove the battery
- Do not dispose in fire
- Keep away from children
- Do not short circuit the charging terminals in the bottom of the headset.


Note: *This series of LoopHEAR™20 products comes in 2 model variations and spare headsets as below. This Instruction Manual covers all of them.*


- 1) LoopHEAR™20 Single Headset System*
- 2) Extra LoopHEAR™20 Headset*

INSTALLATION

(1) Connect the power supply as shown in the diagram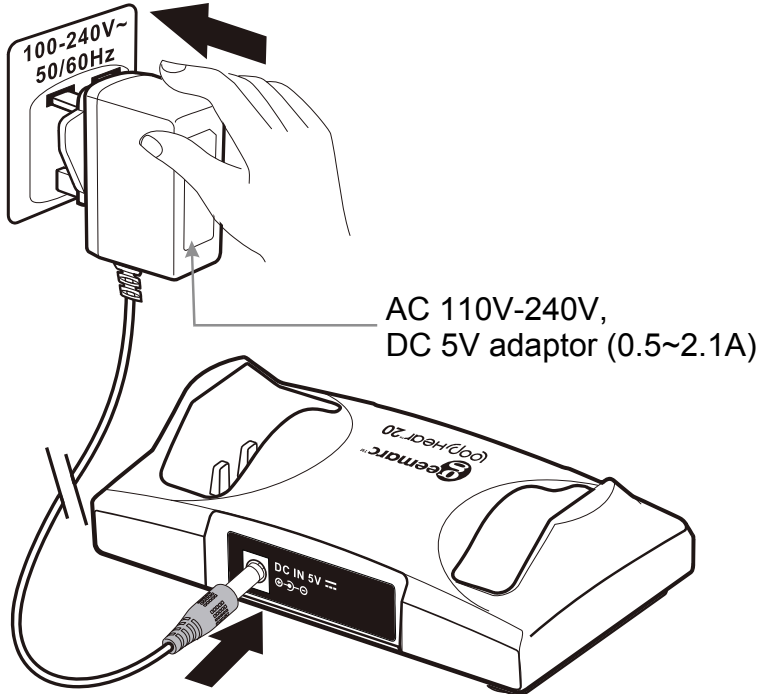


(2) Place the headset in a charging cradle in the Charger base. Verify that the **red** LED charging indicator is ON. When the battery is fully charged a **green** LED turns on.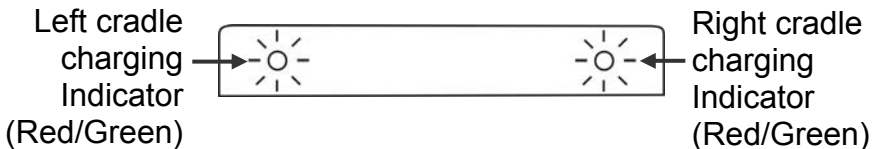

NOTE: It is not possible to overcharge the battery. We recommend that the headset be placed in the charging cradle when not in use.

OPERATION


1. Turn on the headset using the ON/ OFF/ VOLUME (🔌) control and adjust the volume.

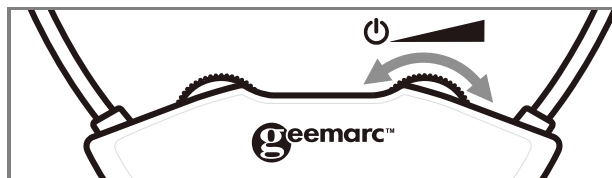2. **Set the mode switch (📶/T) to MIC (📶),** the built in microphone allows you to hear all the surrounding sounds.


3. **Set the mode switch (📶/T) to T (T),** LoopHEAR™20 can be used in rooms especially equipped with a magnetic loop (church/conference room...etc) due to its “built in T-Coil”. LoopHEAR™20 will receive the emitted magnetic waves then amplify them in the LH20. You will receive all the benefits of the “built in T Coil” in the LH20.

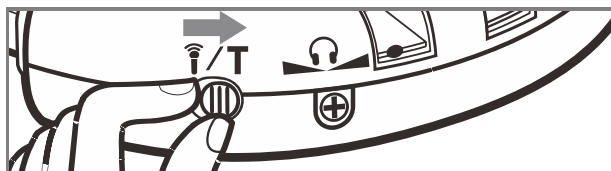4. Use the tone control wheel to adjust the tone (🎵) frequency to suit your needs.

5. Use the included balance (🎧) control tool (👂) to adjust the volume balance between the left and right ears. This control is on the headset.

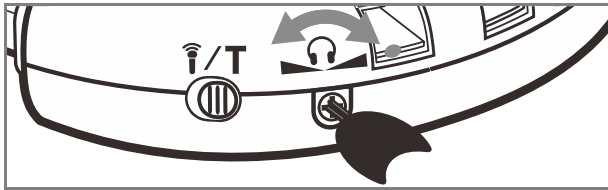6. When you are finished using the system carefully place the headset in the charging cradle making sure that it fits snugly. Check that the charging light turns on.

SAFETY, CARE AND MAINTENANCE

- The system is designed to operate from a 100-240V AC supply and does not have an integral ON/OFF switch. To disconnect the power switch the mains power point switch to OFF or unplug the power supply (AC-DC adapter);
- Clean the headset and ear tips regularly ----- use a cloth for the headset and warm water and a mild detergent for the ear tips (after removal from the headset); (for commercial applications spare ear tips are available for purchase);
- Keep away from heat sources such as hot-plates, and heat generating appliances;
- Keep away from water and other fluids;
- Repairs must be carried out by qualified personnel;
- We suggest that a power surge protector is connected between the mains supply power point and the AC-DC adapter.

PROBLEMS AND SOLUTIONS

The sound is not clear (static/interference)

- Keep away from any windows or interfering device.

The power indicator base's LED lights are not on.

- Check the connection to the wall socket or power outlet.
- Check the connection to the Charger 5V jack.

The charging lights do not come on when the earphone set is in the cradle.

- Realign earphone set; if necessary, move around in the cradle.
- Ensure that the charging cradle is connected to mains power.

General information

Guarantee

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our Helpline or visit our website at www.geemarc.com.

The guarantee does not cover accidents, negligence or breakage to any parts.

The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

Important: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

EEC Declaration

Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/53/UE.

The declaration of conformity may be consulted at www.geemarc.com

Disposal information

Disposal of old electrical & electronic equipment (applicable in the European Union and other European countries with separate collection systems)

The symbol on the product or its packaging indicates that this product may not be treated as household waste. Instead it should be handed over to the applicable collection point for the recycling of electrical and electronic equipment.

By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

For more detailed information about recycling this product, please contact your local council office, your household waste disposal service or the shop where the product was purchased.

Customer support

For product support and help, visit our website at www.geemarc.com

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