**Headset For LoopHEAR™20**  
**Casque audio pour LoopHEAR™20**  
**Kopfhörer für LoopHEAR™20**

English 1

Français 6

Deutsch 11

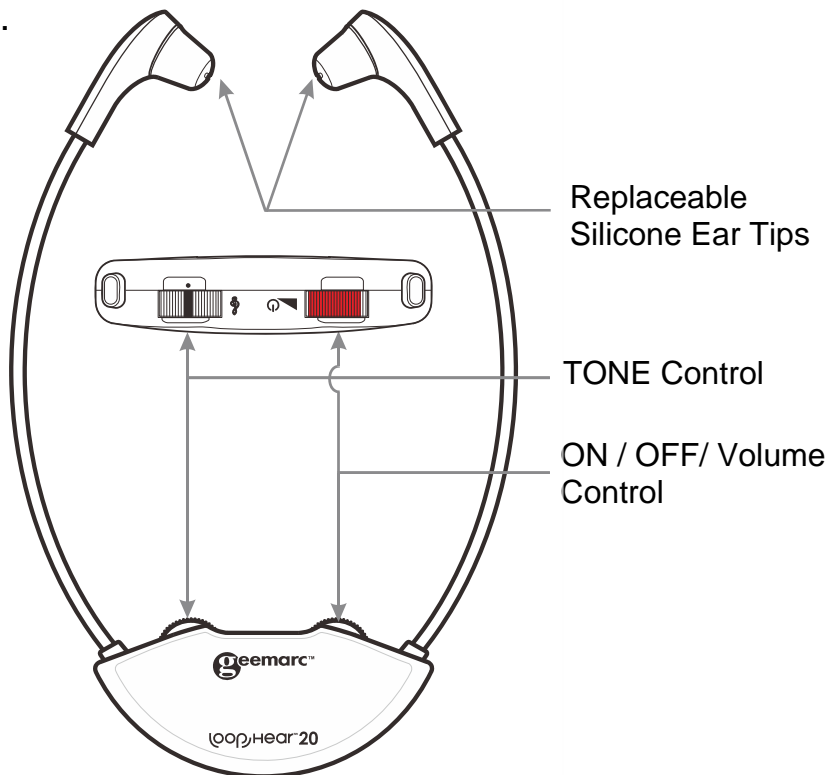
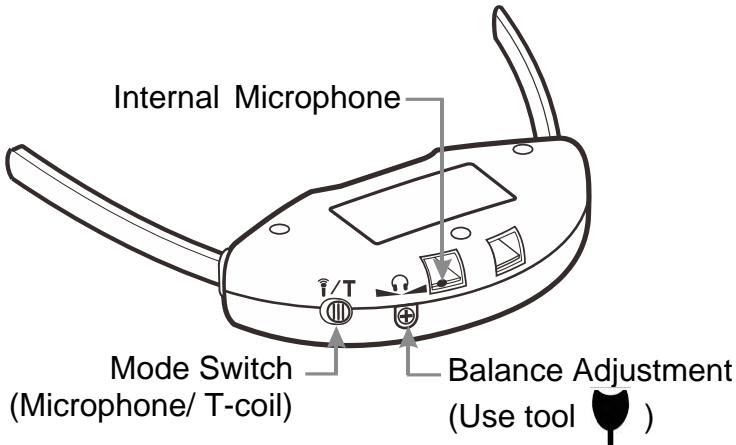
## INTRODUCTION

**LoopHEAR™20AD** is a Personal Sound Amplifier which can be used by people with hearing impairment who do not have “T” coil equipped hearing aids. It amplifies speech together with sounds from a variety of sources like TV, radio and telephone.

The **LoopHEAR™20AD** can be used as an alternative to Hearing Aids.

**WARNING:** This product can greatly amplify the sound. Excessive sound pressure levels can cause hearing damage so it is recommended to initially set the volume to a low level and gradually increase it to a comfortable level.

1. The headset is powered by a 3.7V lithium polymer rechargeable battery which is sealed inside the headset. This type of battery should last for many years and has over 300 recharging cycles.
2. We recommend that the battery be charged for about **2 hours** before the system is used for the first time.
3. Place the headset in a charging cradle in the Charger base. Verify that the **red** LED charging indicator is ON. When the battery is fully charged a **green** LED turns on.

**NOTE:** It is not possible to overcharge the battery. We recommend that the headset be placed in the charging cradle when not in use.

## BATTERY WARNINGS

- Do not attempt to change or remove the battery
- Do not dispose in fire
- Keep away from children
- Do not short circuit the charging terminals in the bottom of the headset.

**NOTE:** Refer to the instruction manual for your main **LoopHEAR™20** for other guidance concerning the use of the headset.

## General information

---

### Guarantee

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our Helpline or visit our website at [www.geemarc.com](http://www.geemarc.com).

The guarantee does not cover accidents, negligence or breakage to any parts.

The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

**Important:** YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

**Please note:** The guarantee applies to the United Kingdom only.

---

### EEC Declaration

Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/53/UE.

*The declaration of conformity may be consulted at [www.geemarc.com](http://www.geemarc.com)*

---

## Disposal information

### Disposal of old electrical & electronic equipment (applicable in the European Union and other European countries with separate collection systems)

The symbol on the product or its packaging indicates that this product may not be treated as household waste. Instead it should be handed over to the applicable collection point for the recycling of electrical and electronic equipment.

By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

For more detailed information about recycling this product, please contact your local council office, your household waste disposal service or the shop where the product was purchased.

---

## Customer support

For product support and help, visit our website at [www.geemarc.com](http://www.geemarc.com)

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