geemarc™ CL7160 Loop

Hear the Difference

- En** 2.3Mhz infrared wireless TV listener Loop system with Optical Connection
- Fr** Casque TV infrarouge à Boucle Magnétique avec entrée analogique, optique et coaxiale
- De** Infrarot TV-Kopfhörer 2,3 MHz mit digitalem, optischem und koaxialem Eingang
- NL** 2,3 MHz infrarode draadloze TV luisteraar volgens het Lussysteem en met optische verbinding
- Esp** Auricular infrarrojo inalámbrico de TV de 2,3 MHz con conexión óptica
- It** Ascoltatore TV infrarosso a 2,3 MHz wireless a collegamento ottico

English P1

Français P15

Deutsch P29

Nederlands P44

Español P57

Italiano P71

INTRODUCTION

The CL7160 is a Personal Sound Amplifier which can be used by people with impaired hearing or normal hearing. It can be used to listen to TV or other audio equipment without disturbing others.

It uses a 2.3MHz Infra-red(IR) system.


CL7160 uses a headset with an induction loop to allow use while wearing a hearing aid which has a T-coil installed. **The hearing aid should be set to the T position for proper operation.**

This product will accept the following audio inputs:

- A) Analog Audio (RCA type output terminals).
- B) Optical Digital Audio.
- C) Coaxial Digital Audio.


WARNING: This product can produce high sound pressure levels in order to assist people with impaired hearing. Excessive sound pressure levels can cause hearing damage so it is recommended to initially set the volume to a low level and gradually increase it to a comfortable level.

PRODUCT CONTENTS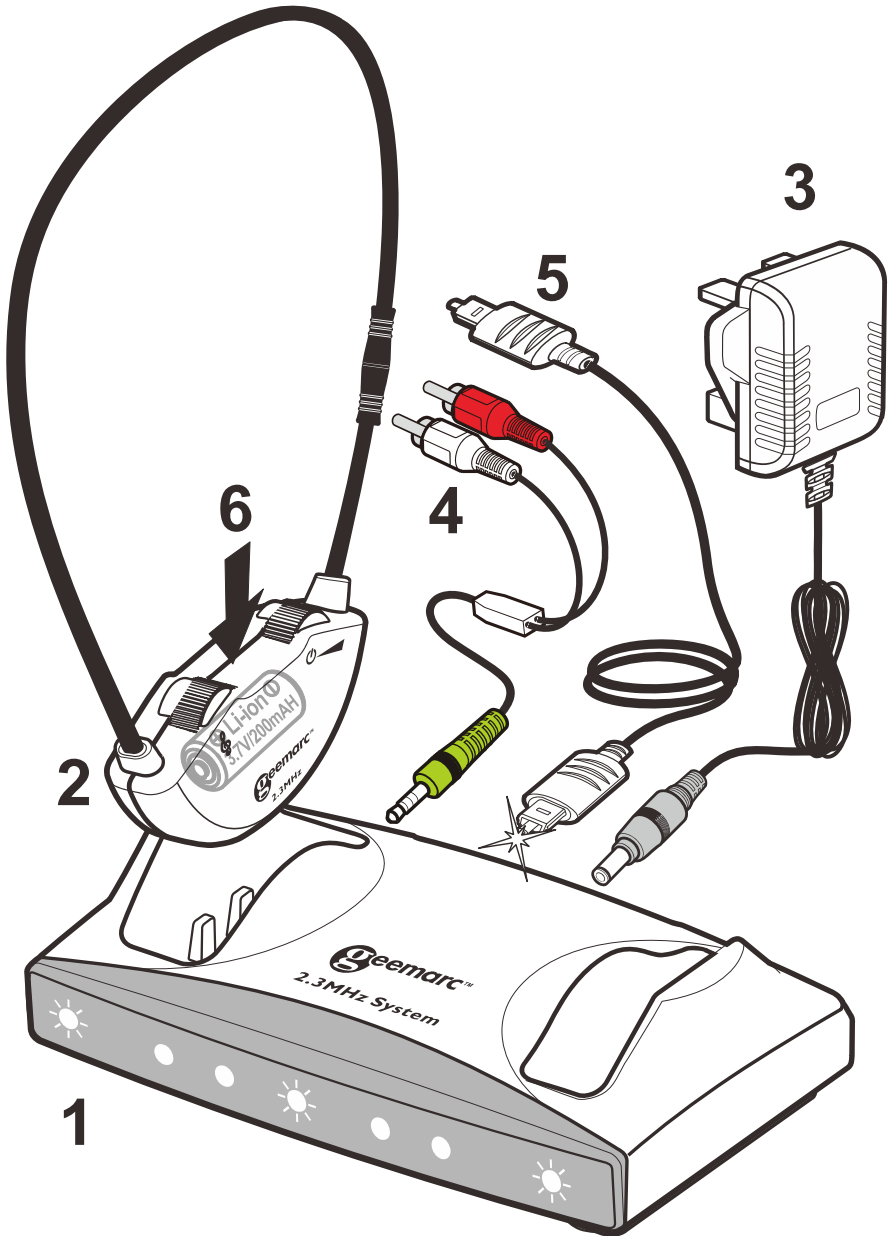


(1) Transmitter / Charger

Front View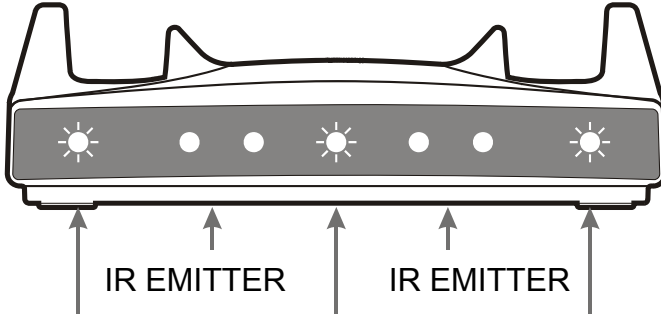


Left Cradle Charging indicator (Red/Green) Power and Emission indicator Right Cradle Charging indicator (Red/Green)

Back View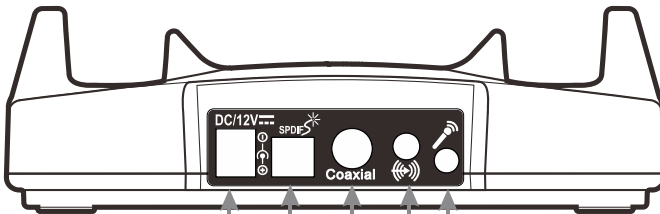


DC 12V Power Jack Digital Audio Optical Input Digital Audio Coaxial Input Microphone Input Jack Analog Audio Input Jack

Bottom View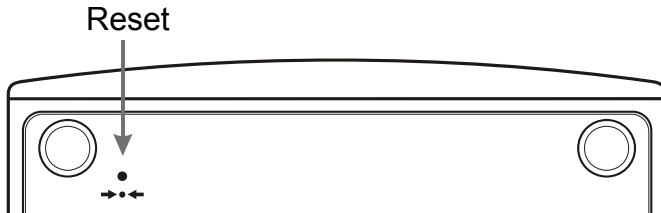

NOTE: The transmitter/charger can charge one or two headsets.

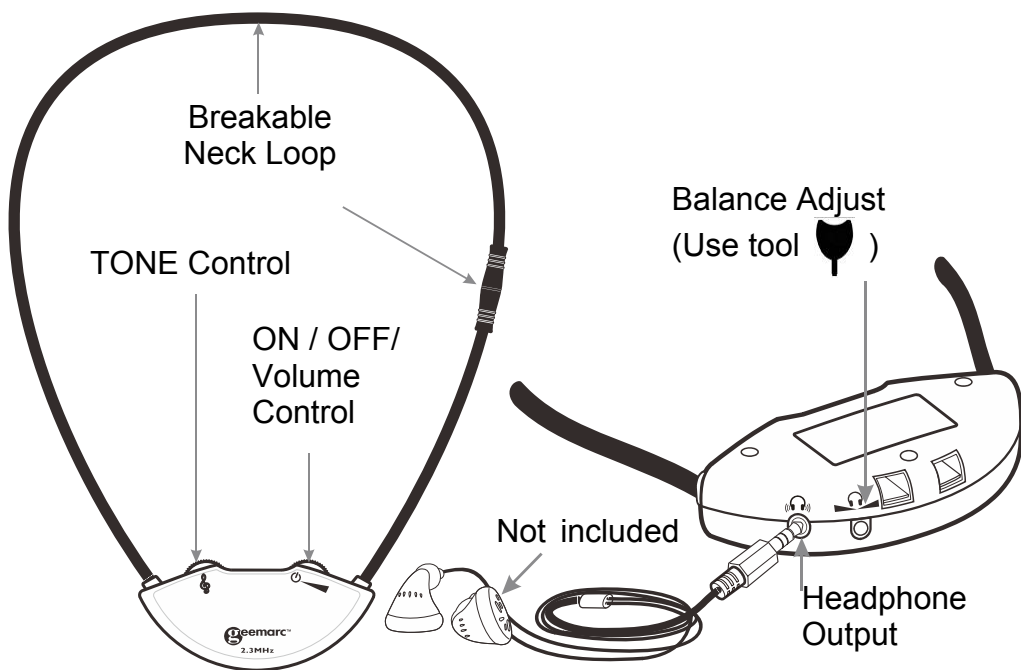(2) Headset

NOTE: Depending on the model purchased there may be either one or two headsets in the box.


(3) Power Supply (AC – DC Adapter, 100 - 240V)**(4) RCA(red/white) to 3.5mm audio cable****(5) Toslink cable (optical digital).**

IMPORTANT: Before using the cable remove the clear plastic protective covers from both ends.

(6) Lithium Polymer Rechargeable battery(Sealed inside the headset) 3.7V/200mAH

NOTE:

1. *This type of battery should last for many years and normally should not need replacing.*
2. *We recommend that the battery be charged for about 2 hours before the system is used for the first time.*
3. *Place the headset in a charging cradle in the Transmitter / Charger base. Verify that the **red** LED charging indicator is ON. When the battery is fully charged a **green** LED turns on.*
4. *If the headset is placed in the charging cradle a **green** LED turns on it means that the headset still has ample charge to operate normally and there is no need to charge it.*

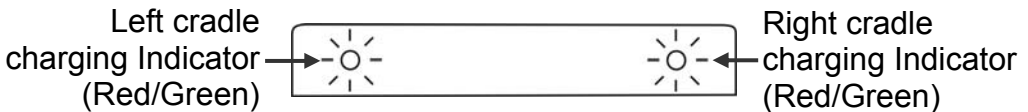5. *It is not possible to overcharge the battery. We recommend that the headset be placed in the charging cradle when not in use.*

BATTERY WARNINGS

- Do not attempt to change or remove the battery
- Do not dispose in fire
- Keep away from children
- Do not short circuit the charging terminals in the bottom of the headset.


Note: *This series of CL7160 products comes in 2 model variations and spare headsets as below. This Instruction Manual covers all of them.*

- 1) *CL7160 Wireless Single Headset System*
- 2) *Extra Wireless Headset*

INSTALLATION

TV SOUND TRANSMITTER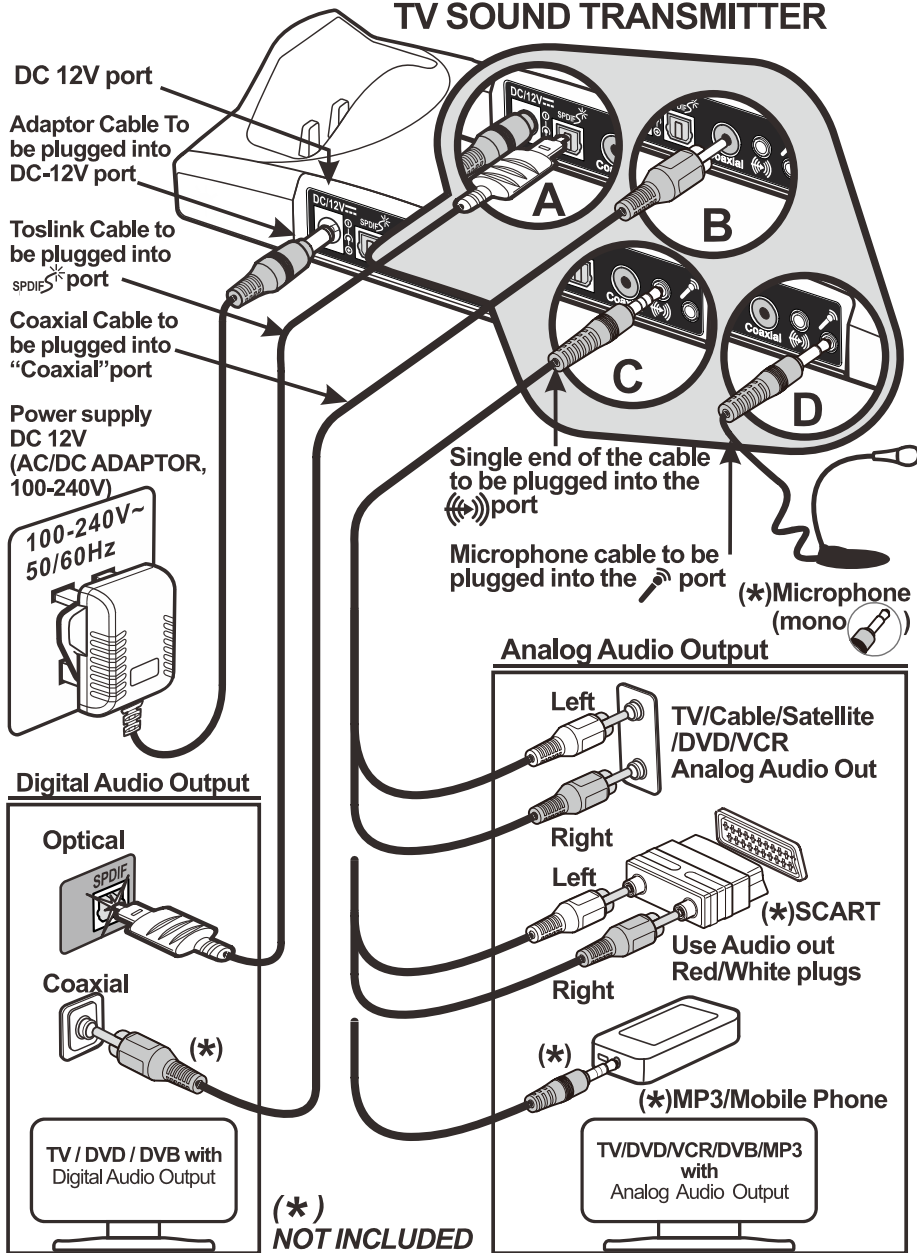

(1) Connect the power supply as shown in the diagram

(2) The transmitter/charger must be connected to the audio **OUTPUT** sockets of the TV, audio equipment, cable or satellite box, DVD or VCR player, etc. When an RCA type cable is used the outputs jacks are red and white. **Refer to the diagram.**

A Scart adapter(not included) may also be used. Use only the red and white terminals.

NOTE: *If the system is connected to a cable or satellite box and to a DVD or VCR player it is necessary to turn off the device which is not in use at the time. When using the DVD or VCR player turn off the cable or satellite box. When watching TV turn off the DVD and VCR player.*

(3) Connecting a microphone

If needed, connect a condenser microphone using the microphone socket located on the rear of the Transmitter /charger.

When connecting to a TV or other equipment which has digital audio output terminals, use either the included Toslink cable (optical digital) or a coaxial cable, depending on which type of output terminals are provided on the equipment. Please refer to the connection diagram on page 6 or to the user manuals of your TV and other equipment.

NOTE: *When using CL7160 with a digital audio input signal the TV audio output must be set to PCM mode. CL7160 is not compatible with Dolby Digital. The PCM setting can normally be done using the audio menu settings of the TV. Please refer to your TV/DVD/Satellite/Cable Box user's manual for instructions for setting to PCM mode.*

NOTE: *If you wish to use the system to listen to TV while the CL7160 is muted so as not to disturb others, the TV audio output must be set to FIXED. Refer to the user manual of your TV for instructions as to how to set to FIXED.*

POSITIONING


The infra-red(IR) signals need a direct line-of-sight connection between the transmitter and the headset. Position the transmitter such that there is a direct line-of-sight connection with the headset when in use. The IR signals may be blocked by objects such as walls, doors, furniture, newspapers, books, etc. The IR diodes are lit during use.

NOTE: *When there is no audio input to the transmitter/charge, transmission is turned off automatically and the blue LED goes off. This is resumed automatically when audio signal is restored.*

OPERATION

1. Make sure that the transmitter/charger is correctly connected to the mains supply using the adapter supplied.
2. Put on the headset such that the front(silkscreened) part of the headset is facing the transmitter.
3. Turn on the headset using the ON/ OFF/ VOLUME  control and adjust the volume.
4. Use the tone control wheel to adjust the tone  frequency to suit your needs.
5. If you wish to listen to TV while not wearing a hearing aid you may connect standard headphones which have a 3.5mm plug by inserting the 3.5mm plug into the 3.5mm audio output socket in the headset.
Use the included balance control tool () to adjust the volume balance between the left and right ears. This control is on the headset  .
6. When you are finished using the system carefully place the headset in the charging cradle making sure that it fits snugly. Check that the charging light turns on.

NOTE: The induction neck loop has an "easy-break" plug and socket (refer to the sketch on page 4). Should the loop become tangled or you are having difficulty removing the loop from around your neck simply pull on the loop near the easy-break and it will come free easily.

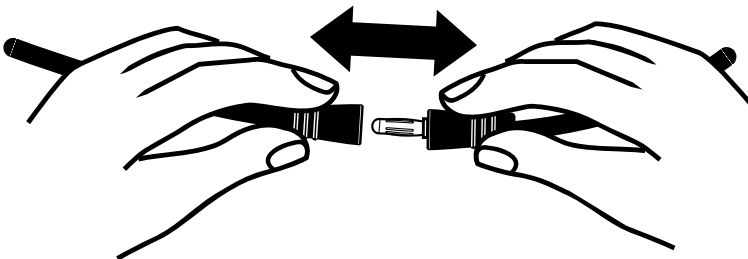SAFETY, CARE AND MAINTENANCE

- The system is designed to operate from a 100-240V AC supply and does not have an integral on/off switch. To disconnect the power switch the mains power point switch to OFF or unplug the power supply (AC-DC adapter);
- Keep away from heat sources such as hot-plates, and heat generating appliances;
- Keep away from water and other fluids;
- Repairs must be carried out by qualified personnel;
- We suggest that a power surge protector is connected between the mains supply power point and the AC-DC adapter.

PROBLEMS AND SOLUTIONS

The sound is not clear (static/interference)

- Check the line-of-sight connection.
- Make sure that the Blue LEDs is lit.
- Check the transmitter location (height above the floor).
- Keep away from any windows or interfering device.
- Check if the audio cable is connected to an analogue (RCA) audio-OUT jack on TV, cable or satellite decoder /receiver.
- Check that the audio cable is NOT connected to a digital audio-out jack.
- The power outlet may already be in use with another device.
- Audio cable is connected to the transmitter's audio jack.

The transmitter's LED lights are not on.

- Check the connection to the wall socket or power outlet.
- Check the connection to the CL7160 12V jack.

The charging lights do not come on when the earphone set is in the cradle.

- Realign earphone set; if necessary, move around in the cradle.
- Ensure that the charging cradle is connected to mains power.

PRODUCT SPECIFICATIONS

General data :

Frequency: 2.3MHz

Modulation: Infrared, mono

Approx. range: 55 sq.m (approx. 600 sq. ft.)

Audio frequency response: 100-16,000 Hz

Total harmonic distortion: < 1%

Signal to noise Ratio: 70 dB

Auto ON/OFF

Headset :

Battery requirement: Lithium Polymer Rechargeable battery, 3.7V / 200mAH

Operating time: Max. 6 hrs.

Weight: approx. 46 – 50 g

Transmitter :

Audio connection: 3.5mm input jack/RCA type cable/optical digital/coaxial digital/microphone input.

Power Supply: DC Power Supply 12V, 500mA

Weight: Approx. 180 g

Dimensions (H x W x D): 4.8 x 10.5 x 15.4 cm

RESET SWITCH: Sometimes computers malfunction due to electrostatic discharge or other electrical disturbances, or when DC Jack is plugged. If this happens push a thin rod into the RESET hole  in the base of transmitter unit and press down for about one second. This Resets the transmitter unit, clears its memory and returns it to normal operation.

General information

Guarantee

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our Helpline or visit our website at www.geemarc.com.

The guarantee does not cover accidents, negligence or breakage to any parts.

The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

Important: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

EEC Declaration

Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5 EEC.

The declaration of conformity may be consulted at www.geemarc.com

This Product has been tested by an accredited testing laboratory and is in compliance with the following standards and directives:

- EN60065
- EMC 2004/108/EC
- RoHS
- UL1642

Disposal information

Disposal of old electrical & electronic equipment (applicable in the European Union and other European countries with separate collection systems)

The symbol on the product or its packaging indicates that this product may not be treated as household waste. Instead it should be handed over to the applicable collection point for the recycling of electrical and electronic equipment.

By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

For more detailed information about recycling this product, please contact your local council office, your household waste disposal service or the shop where the product was purchased.

Customer support

For product support and help, visit our website at

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