Plug in portable telephone amplifier, with volume and tone adjustment

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English Page 2

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Deutsch Page 18

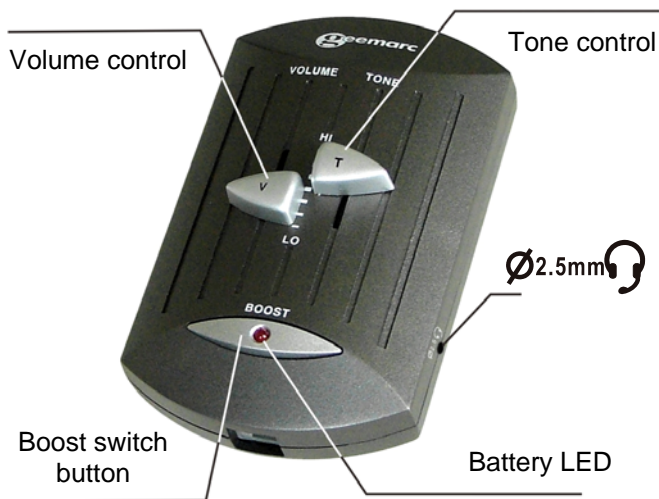
CAUTION: THIS PRODUCT IS SPECIFICALLY DESIGNED FOR PEOPLE WITH HEARING DIFFICULTIES. IT HAS VOLUME AND TONE CONTROLS AND WHEN ATTACHED TO YOUR TELEPHONE IT WILL ALLOW YOU TO ADJUST YOUR TELEPHONE ACCORDING TO YOUR INDIVIDUAL REQUIREMENTS. DUE CARE MUST BE TAKEN BY ALL USERS TO ENSURE THAT THE VOLUME CONTROL IS SET TO A COMFORTABLE LEVEL FOR EACH USER. CARE SHOULD BE TAKEN EXERCISED TO ENSURE THAT ANY NEW USERS ARE FAMILIAR WITH THIS REQUIREMENT.

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DESCRIPTION CLA40VOX	3
INSTALLATION	5
SETTINGS	6
Volume control	6
Tone control	6
Boost switch	6
Battery LED	6
TROUBLE SHOOTING	7
GUARANTEE	8
WALL MOUNT TEMPLATE	26
	27


TOP

REAR

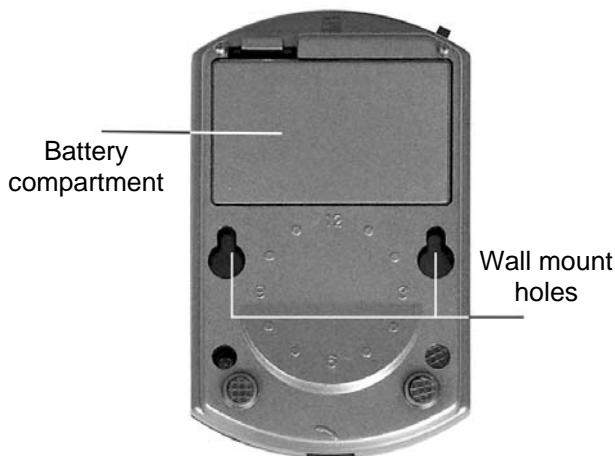BOTTOM

**FRONT**

(*) Output socket connection (2)

(*) Classified TNV-3 according to EN60950 standard.

HOW TO SET THE BATTERY

The CLA40VOX amplifier needs with a 9V PP3 battery (not provided). To insert the battery into the amplifier:

- Open the battery compartment located on the base by pressing the small plastic lock.
- Insert the 9V battery with the correct polarity into the connector in the battery compartment.
- Replace the battery cover.
- The amplifier is now ready to be connected to the phone.

HOW TO CONNECT THE CAL40VOX TO THE PHONE

To connect your phone to the CLA40VOX amplifier:

- Unplug the handset from your telephone base socket.
- Connect the small lead supplied with the CLA40VOX to the telephone base socket and to the input socket (1) located at the top of your CLA40VOX .
- Connect your handset curly cord to the output socket (2) located at the front of your CLA40VOX.

VOICE CONTROL : As soon as the CLA40VOX will detect a voice, a sound from the earpiece or the microphone, it will turn ON automatically (LED lit) then during a silence more than 15secs it will also turn OFF automatically.

IMPORTANT : The CLA40VOX cannot be used with one-piece telephones which have a keypad on the handset.

The CLA40VOX can only be used with standard phones without any amplification or tone adjustment.

VOLUME CONTROL

You can adjust the volume control level by using the "**VOLUME**" control slide switch located on the top. Volume can be adjusted from the standard level of your phone up to 30 dB or more.

TONE CONTROL

You can adjust the tone by using the 5 position "**TONE**" switch located on the top. The tone can be adjusted in 5 dB step from the 0dB mid point to the **Hi** or **LO** positions shown according to your individual requirements.

BOOST SWITCH

If you wish you can increase the receiving up to 10 dB more by pressing and holding the Boost switch button. During this operation the microphone is muted to avoid feedback problems. To speak release the button so that other party can hear you.

BATTERY INDICATOR

The battery LED indicator should illuminate every time you use your CLA40VOX. If the LED start to fade or does not light when your CLA40VOX is in use, you should change the battery to ensure optimum sound quality and performance.

IMPORTANT: Always disconnect CLA40VOX from the phone before replacing the battery.

IF THE AMPLIFIER DOES NOT WORK

- Make sure all connections are plugged into the correct sockets (phone line cord, amplifier connections).
- Make sure battery is correctly inserted and is fully charged.

If you still have a problem when using your amplifier follow the instructions below:

- Disconnect the amplifier from the telephone.
- Connect the handset to the telephone.
- If the problem ceases, the amplifier may be faulty or your telephone socket wiring may not be compatible with the CLA40VOX. Call our HELPLINE for advice.
- If you have a problem after disconnecting the amplifier, the problem may come from your telephone or your telephone line socket.

In this case you will have to inform your telephone company about the problem or take your telephone to a repair centre.

For product support and help visit our website at

www.geemarc.com

Telephone: +44 (0)1707 384438

or Fax +44 (0)1707 832529

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our helpline or visit our website at **www.geemarc.com**.

The guarantee does not cover accidents, negligence or breakages to any parts.

The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

The guarantee applies to the United Kingdom only.

Declaration: Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/ EEC and in particular article 3 section 1a, 1b and section 2.

The declaration of conformity may be consulted at www.geemarc.com

Recycling directives: The WEEE (Waste Electrical and Electronic Equipment) Directive has been put in place so that products at the end of life are recycled in the best way.

When this product is out of order, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip.

Deposit the product in an appropriate WEEE skip.

- Or hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

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