CLA-30/40

GB User guide index p.2

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CAUTION: THIS PRODUCT IS SPECIFICALLY DESIGNED FOR PEOPLE WITH HEARING DIFFICULTIES. IT HAS VOLUME AND TONE CONTROLS AND WHEN ATTACHED TO YOUR TELEPHONE IT WILL ALLOW YOU TO ADJUST YOUR TELEPHONE ACCORDING TO YOUR INDIVIDUAL REQUIREMENTS. DUE CARE MUST BE TAKEN BY ALL USERS TO ENSURE THAT THE VOLUME CONTROL IS SET TO A COMFORTABLE LEVEL FOR EACH USER. CARE SHOULD BE TAKEN EXERCISED TO ENSURE THAT ANY NEW USERS ARE FAMILIAR WITH THIS REQUIREMENT.

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IN-LINE

INDEX

DESCRIPTION CLA-30/40	4
INSTALLATION	8
SETTINGS	9
Volume control	9
Tone control	9
Boost switch (CLA-40 only)	9
Battery LED	9
TROUBLE SHOOTING	10
GUARANTEE	11
WALL MOUNT TEMPLATE	16

For product support and help
visit our website at www.geemarc.com
telephone 01707 384438
or fax 01707 372529

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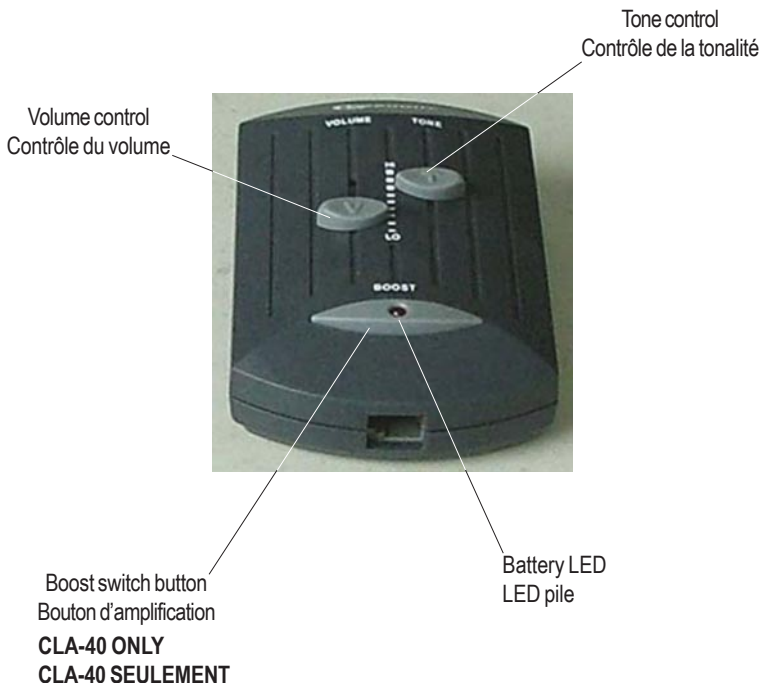
DESCRIPTION CLA-30/40

BOTTOM DESSOUS

Battery compartment
Compartiment pile

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INSTALLATION

1 - HOW TO SET THE BATTERY

The CLA-30/40 amplifier needs with a 9V PP3 battery (not provided).

To insert the battery into the amplifier:

- Open the battery compartment located on the base by pressing the small plastic lock.
- Insert the 9V battery with the correct polarity into the connector in the battery compartment.
- Replace the battery cover.
- The amplifier is now ready to be connected to the phone.

2 - HOW TO CONNECT THE CLA-30/40 TO THE PHONE

To connect your phone to the CLA-30/40 amplifier:

- Unplug the handset from your telephone base socket.
- Connect the small lead supplied with the CLA-30/40 to the telephone base socket and to the input socket (1) located at the top of your CLA-30/40 .
- Connect your handset curly cord to the output socket (2) located at the front of your CLA-30/40.
- After a few seconds the battery LED will light.

IMPORTANT:

If you don't have dial tone, you will have to use the twelve position switch located on the bottom. This switch is used to match the CLA-30/40 to the connections in your handset because not all telephone handset sockets have the same wiring. If you cannot hear the dial tone or the microphone does not work properly, follow the instructions below:

1. Replace the handset on the cradle and off hook again.
2. Turn the rotary switch position until get the dial tone in the earpiece.
3. To test that the microphone is connected correctly, blow lightly into the

SETTINGS

mouthpiece. If you hear the 'blowing sound' in the earpiece, your CLA-30/40 is now set. If you cannot hear the 'blowing sound', repeat 2. and 3. until both the earpiece and microphone are working correctly.

Note 1: The rotary switch is factory set in the most of common wiring position so always try this first before turning the switch.

Note 2: The rotary switch has deliberately been made rather stiff to avoid misoperation or accidental change of setting. We apologise if you find this inconvenient but after first set-up, the switch will not be to turned again unless you change your phone.

Note 3: The CLA-30/40 cannot be used with one-piece telephones which have a keypad on the handset.

CLA-30/40 is now ready to use.

3 - VOLUME CONTROL

You can adjust the volume control level by using the "VOLUME" control slide switch located on the top. Volume can be adjusted from the standard level of your phone up to 30 dB or more.

4 - TONE CONTROL

You can adjust the tone by using the 5 position "TONE" switch located on the top. The tone can be adjusted in 5 dB step from the 0dB mid point to the Hi or LO positions shown according to your individual requirements .

5 - BOOST SWITCH (CLA-40 ONLY)

If you wish you can increase the receiving up to 10 dB more by pressing and holding the Boost switch button. During this operation the microphone is muted to avoid feedback problems. To speak release the button so that other party can hear you.

TROUBLE SHOOTING

6 - BATTERY INDICATOR

The battery LED indicator should illuminate every time you use your CLA-30/40. If the LED starts to fade or does not light when your CLA-30/40 is in use, you should change the battery to ensure optimum sound quality and performance.

IMPORTANT: Always disconnect CLA-30/40 from the phone before replacing the battery.

IF THE AMPLIFIER DOES NOT WORK

- Make sure all connections are plugged into the correct sockets (phone line cord, amplifier connections) and try other position on the rotary switch.

- Make sure battery is correctly inserted and is fully charged.

If you still have a problem when using your amplifier follow the instructions below:

- Disconnect the amplifier from the telephone.

- Connect the handset to the telephone.

- If the problem ceases, the amplifier may be faulty or your telephone socket wiring may not be compatible with the CLA-30/40. Call our HELPLINE for advice.

- If you have a problem after disconnecting the amplifier, the problem may come from your telephone or your telephone line socket.

In this case you will have to inform your telephone company about the problem or take your telephone to a repair centre.

IMPORTANT: The CLA-30/40 can not be used with telephones which include a keypad on the handset. The CLA-30/40 can only be used with standard phones without any amplification or tone adjustment.

GUARANTEE

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc will guarantee it for the period of one year. During this time, all repairs or replacements are free of charge. Should you experience a problem then contact our helpline or visit our website at www.geemarc.com. The guarantee does not cover accidents, negligence or breakages of any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM

Please note: The guarantee applies to the United Kingdom only.

Declaration : Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC and in particular article 3 section 1a, 1b and section 3.

Telephone connection : Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN60950 standard.

Recycling directives : The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products in end of life are recycled in the best way.

When this product is out of order, please do not put it in your domestic waste bin.

Please use one of the following disposal options :

- Remove the batteries and deposit them in an appropriate WEEE skip.

Deposit the product in in a appropriate WEEE skip.

- Or hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help
visit our website at www.geemarc.com
telephone 01707 384438
or fax 01707 372529

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