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
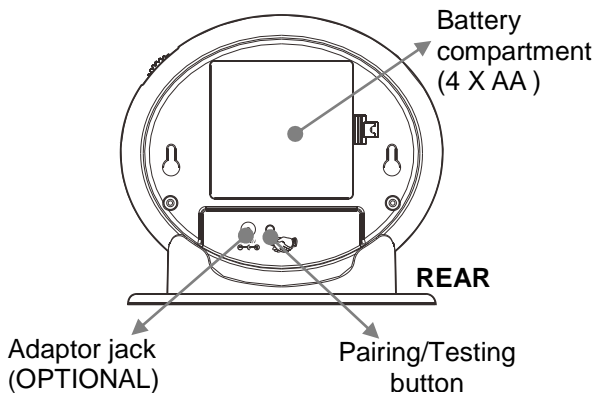
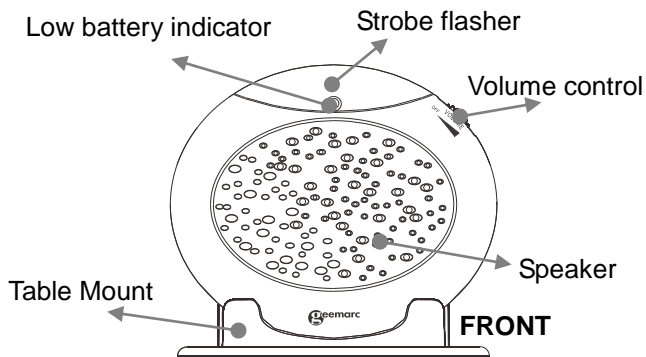
# INSTALLATION
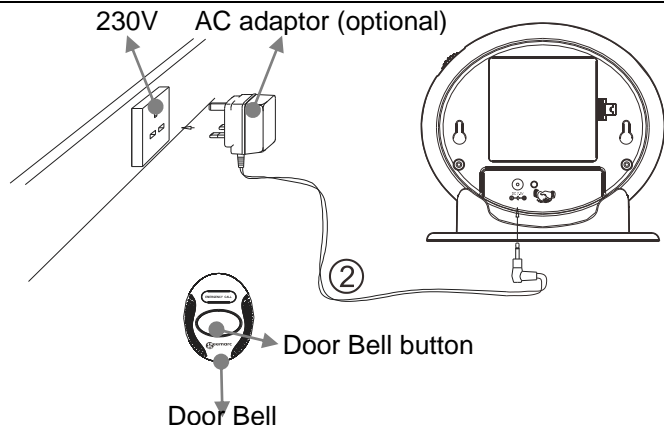
## Installation (See drawing)

- 1 – Insert the 23A 12V battery into the door bell.
- 2 – Insert the 4 (AA) batteries into the battery compartment main unit or connect the adaptor 7.5V DC/1A (optional).


**Electrical connection :** The apparatus is designed to operate from a 230V 50Hz supply only. (Classified as «hazardous voltage» according to EN60950 standard). The apparatus does not incorporate an integral power on/off switch. To disconnect the power, either switch off supply at the mains power socket or unplug the AC adaptor. When installing the apparatus, ensure that the mains power socket is readily accessible.

# INSTALLATION
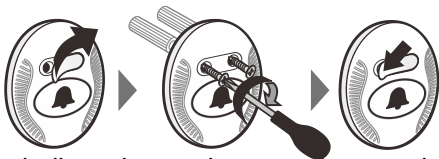
## Setting up the Door Bell

---

1. The door bell can be carried around by people who need assistance
2. The door bell can be fixed on the wall or a door frame. Drill two holes horizontally, 14mm apart. Insert the two supplied anchors into the two holes. Hang the door bell onto the two anchors and then insert and tighten the screws. Finally, stick the blank sticker over the two holes.

3. The door bell can be stuck onto a door or window using the supplied strong adhesive tape. Remove the protective paper from the adhesive tape on the back of the door bell and then stick to the required door, window or surface.

## Wall Mounting the Chime Unit

---

1. Drill two holes horizontally, 96mm apart, and leave the screws protruding from the wall by 5mm.
2. Hang the base on to the two screws and pull the base down to lock it.

## Table Mounting

---

If you want to mount the unit on the table simply place the unit into the base cradle, and then place the base cradle on the table.

# INSTALLATION

## Change the Battery of the Indoor Unit (optional)

---

The unit requires 4xAA batteries

1. Open the battery door.
2. Insert four AA batteries into the compartment.
3. Snap the battery compartment door back into place.

**Note:** When the low battery indicator keeps flashing, it is time to replace the batteries. Alternatively, use a mains adaptor as an option (contact **01707 387602**).

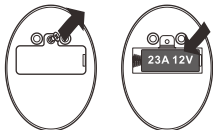
## Install and Change the Battery of the Door bell

---

Install the 23A 12V battery into the door bell

To replace the battery, unscrew the door bell from the wall, and unscrew the one screw located on the back to gain access to the battery compartment. Replace the battery then replace the battery compartment flap and screw and remount.

**Note:** Do not use a rechargeable battery. Do not put the used battery into the household rubbish, pay attention to regulations regarding battery disposal.

# INSTALLATION

## Volume Control Slide Switch

---

The volume control slide switch can be used to set the volume of the alarm sound and switch the unit on/off.

## The Pairing Procedure:

---

1. Press and hold the Pairing/Test button  until the unit makes a beep sound, then release the Pairing/Test button.
2. Press and hold the Pairing/Test button  again, then press and hold the remote button at the same time until the strobe flasher flashes. The unit makes a beep sound to confirm pairing has been successful.

Please note that the pairing procedure overwrites the last doorbell that was paired to the unit.

To check whether the device is working properly and has been paired correctly, press the pairing/testing button once. The device should sound the alarm and flash.

## Operation

---

Once the Amplicall 16 and the door bell have been placed in the required locations and all required batteries have been inserted, you are ready to use the device. To use the device, the person who needs assistance must press the door bell button of the door bell.

# TROUBLESHOOTING

## Trouble shooting

<b>Symptom</b>	<b>Solution</b>
The unit makes the “dingdong” sound (ad hoc or continuously) without the door bell being pressed	<ol style="list-style-type: none"><li>1) Check the batteries are inserted correctly</li><li>2) Check the power adaptor is inserted into the unit and the wall correctly (if using mains power)</li><li>3) Turn off the power switch for 10 minutes and then turn it on</li></ol>
Low battery LED is flashing.	<ol style="list-style-type: none"><li>1) Turn off the power switch for 10 minutes and then turn it on</li><li>2) Insert new batteries into the base unit</li></ol>
Batteries and power adaptor are connected correctly, yet no “Dingdong” sound when the door bell is pressed.	<ol style="list-style-type: none"><li>1) Turn off the power switch for 10 minutes and then turn it on</li><li>2) Replace the base unit batteries</li><li>3) Replace the door bell batteries</li></ol>

# GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at **[www.geemarc.com](http://www.geemarc.com)**. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

**IMPORTANT:** YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

**Please note :** The guarantee applies to the United Kingdom only.

**Declaration :** Hereby Geemarc Telecom SA declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/53/UE and in particular article 3 section 1a, 1b and section 2.

*The declaration of conformity may be consulted at **[www.geemarc.com](http://www.geemarc.com)***

# GUARANTEE

**Recycling directives** : The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way. When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it. Thus if you respect these instructions you ensure human health and environmental protection.

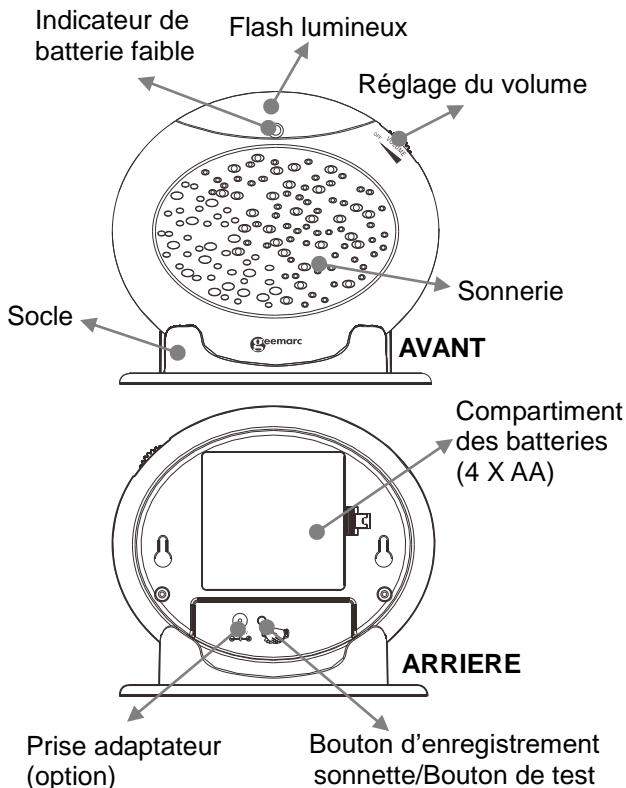
For product support and help visit our website  
at [www.geemarc.com](http://www.geemarc.com)  
Telephone **01707 387602**  
Or fax **01707 832529**

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