geemarc™
Hear the Difference

CL7200

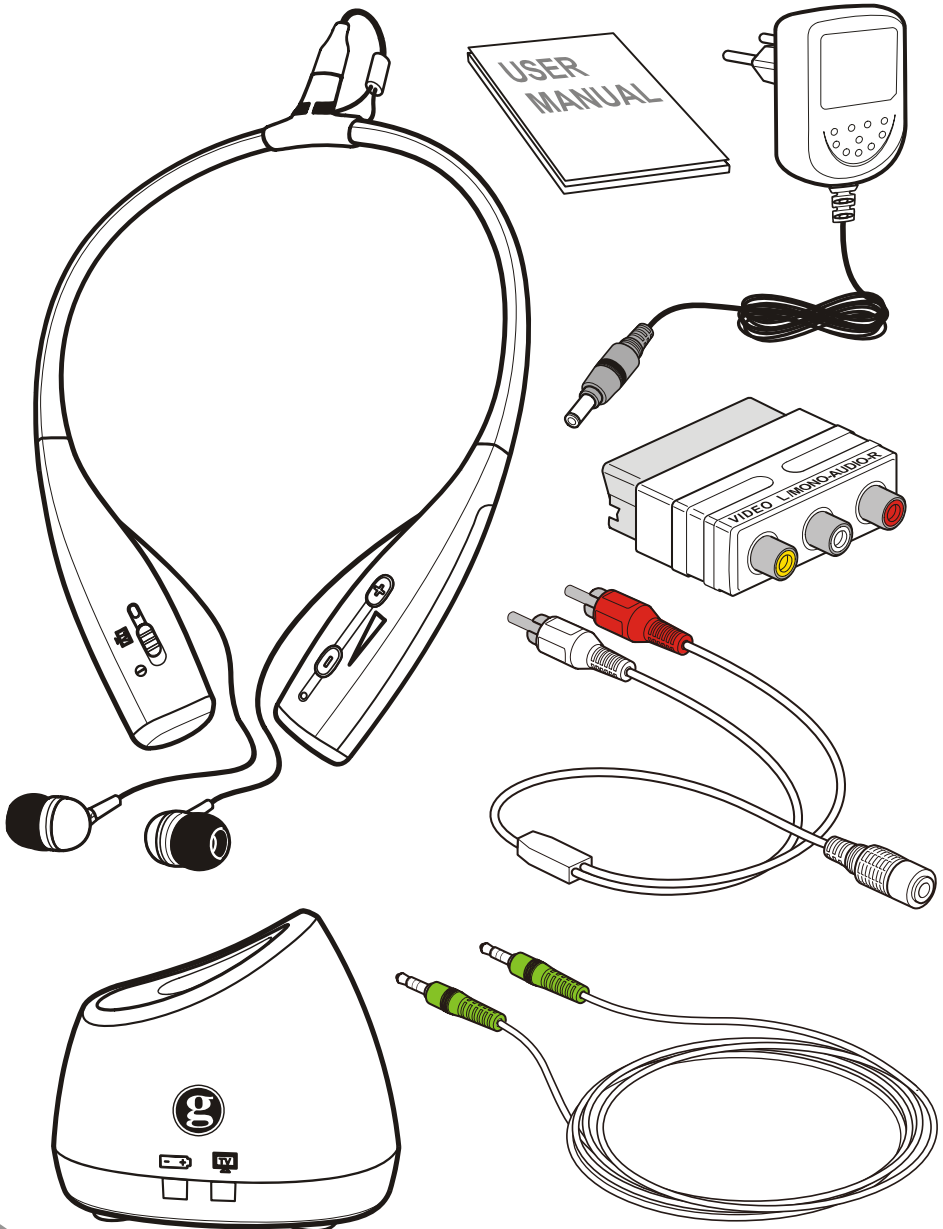
En	Digital TV Listener
Fr	Casque TV Numérique
D	Digitaler TV-Kopfhörer
Español	Oyente activo de TV digital
Italiano	Cuffia per TV Digitale
Nederlands	Digitaal TV Luisterapparaat
עברית	מאזין דיגיטלי לטלוויזיה
Svenska	Digital TV-lyssnare

En Unpacking
Fr Contenu du Colis
De Auspacken
Esp Desempaque

It Estrarre dall'imbaggio
NL Uitpakken
עברית פריקה
Svenska Packa upp


a


En Installation
Fr Installation
De Installation
Esp Instalación

It Installazione
NL Installatie
עברית התקנה
Svenska Installation

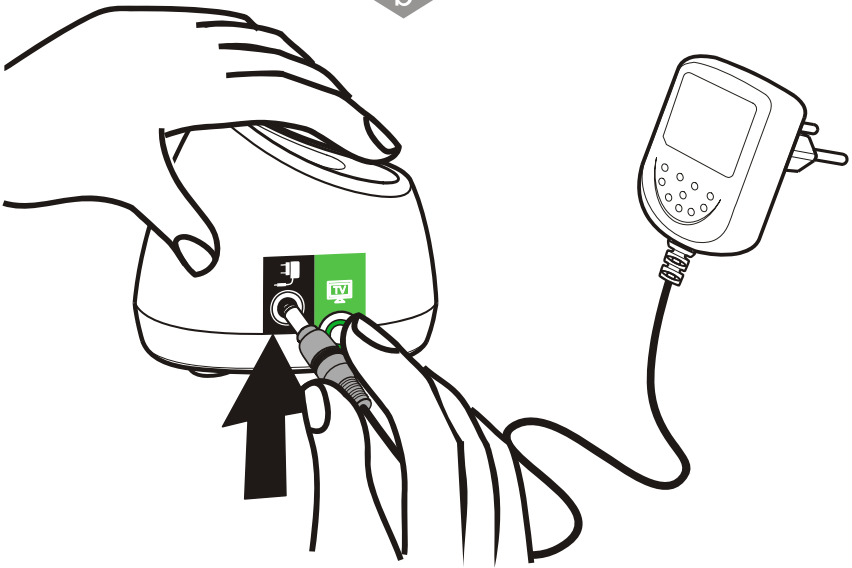b


c

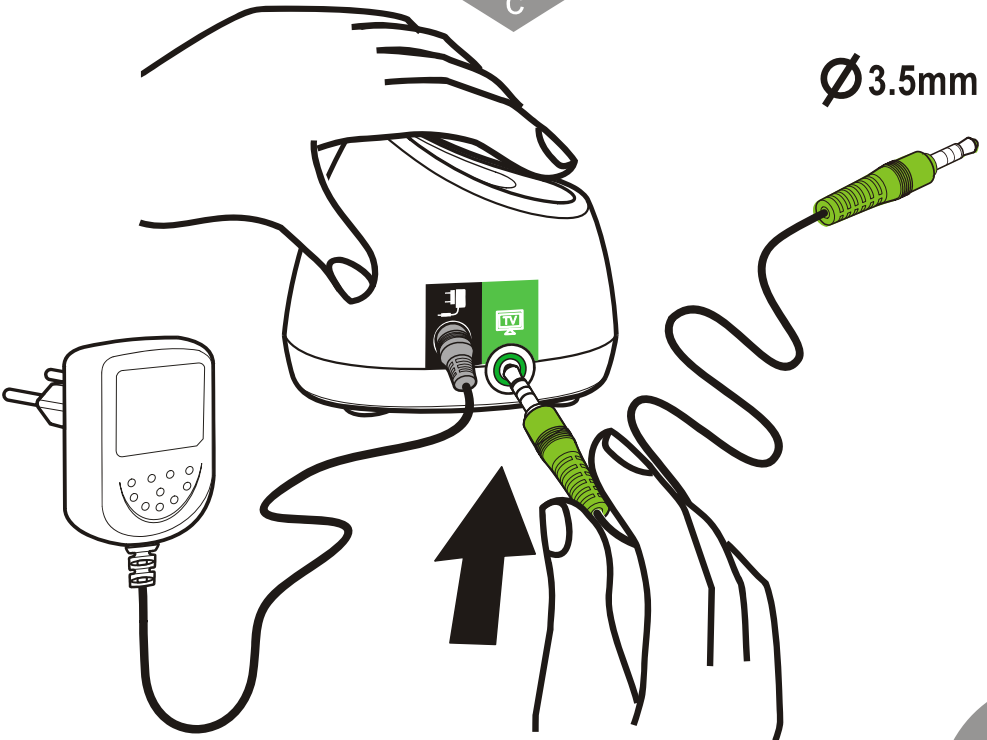
En	Installation	It	Installazione
Fr	Installation	NL	Installatie
De	Installation	עברית	התקנה
Esp	Instalación	Svenska	Installation

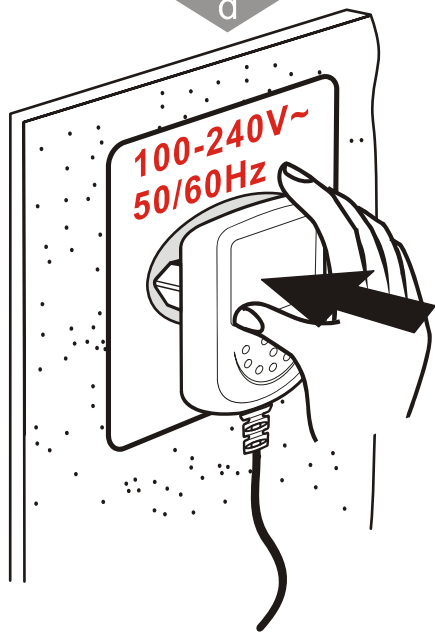
d

e

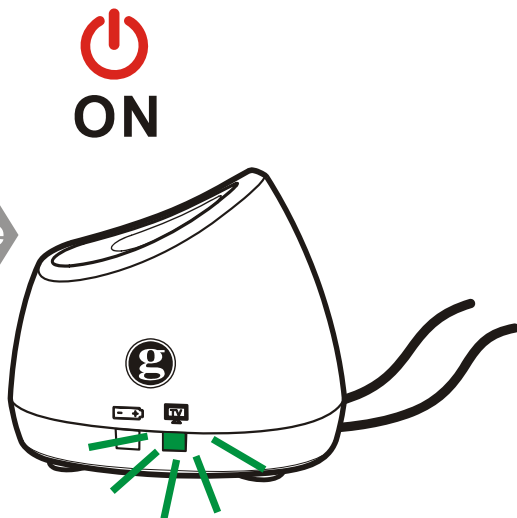
f

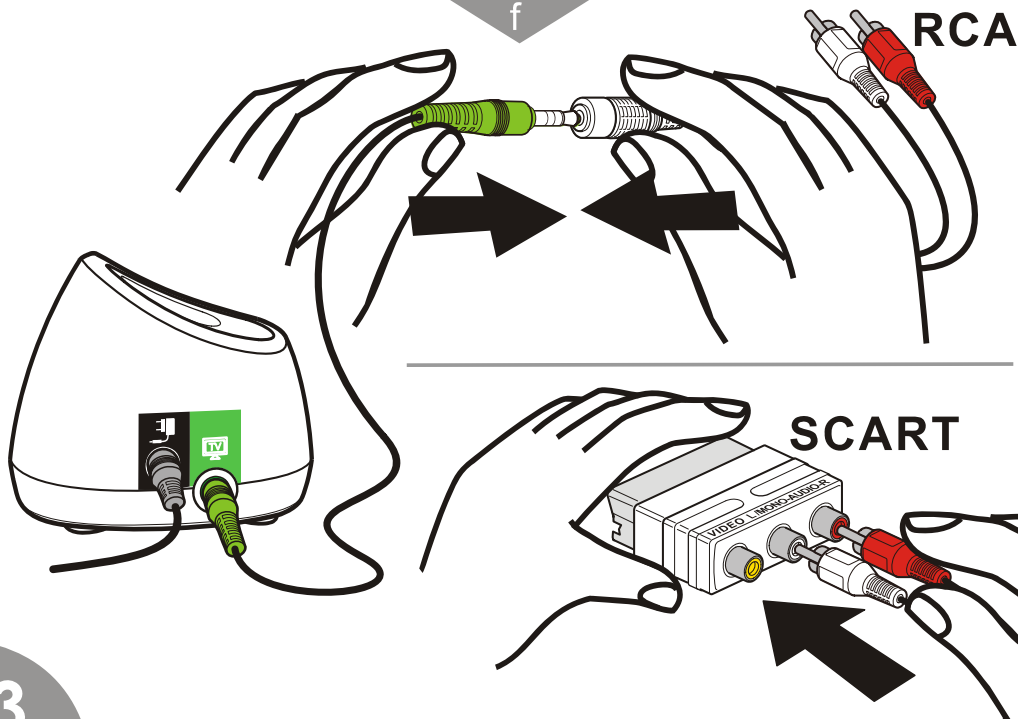
En Installation
 Fr Installation
 De Installation
 Esp Instalación

It Installazione
 NL Installatie
 עברית התקנה
 Svenska Installation

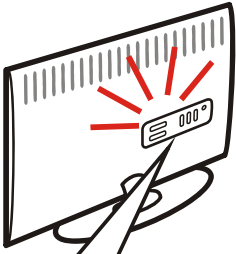g

TYPE A

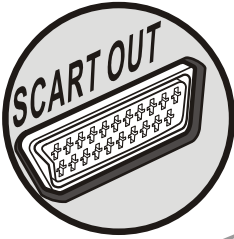
TYPE B


TYPE C

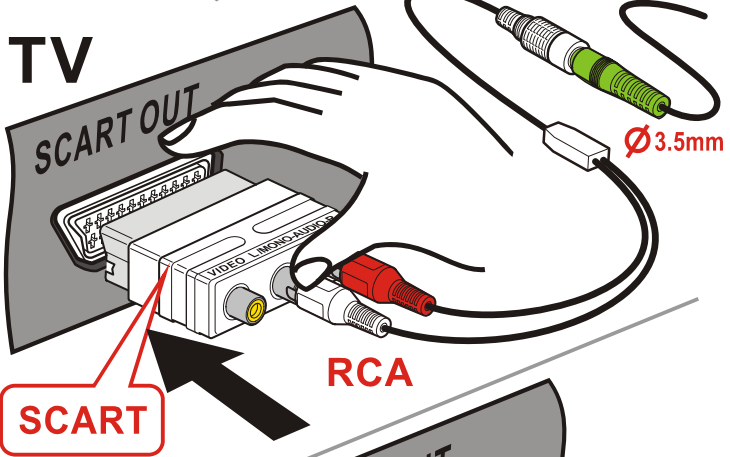
TV

TV

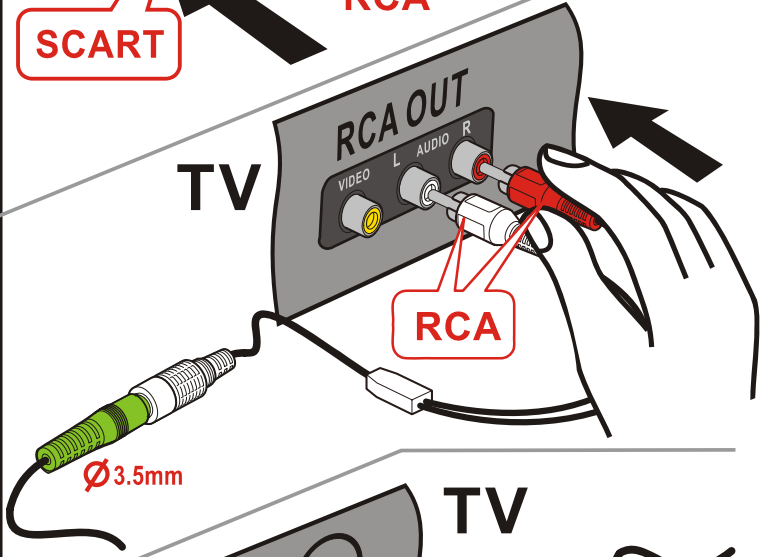
TV

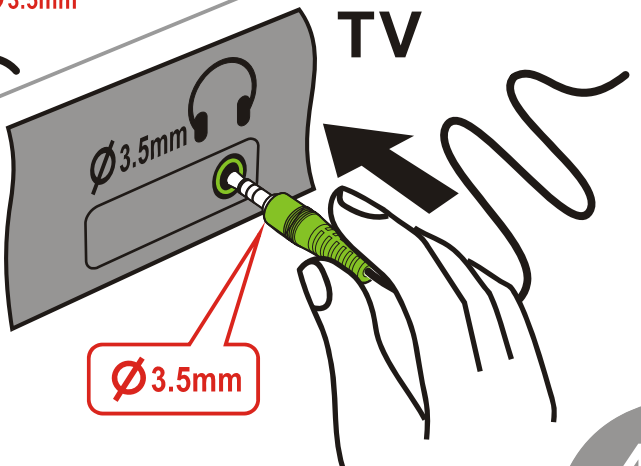
En	Charging	It	Ricarica
Fr	Chargement	NL	Opladen
De	Aufladen	עברית	טעינה
Esp	Carga	Svenska	Laddar

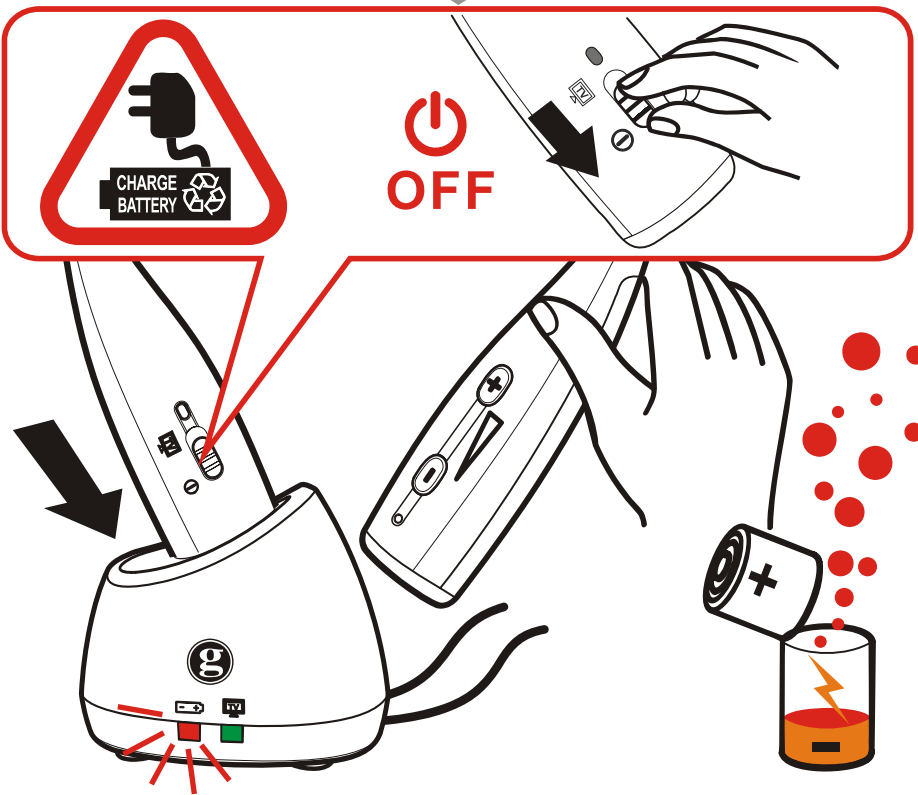h


i

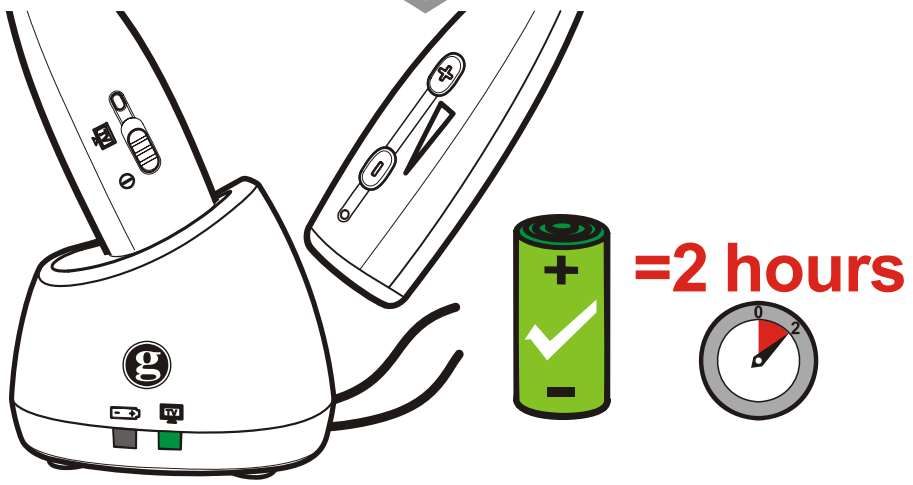En	Pairing
Fr	Enregistrement
De	Anmelden
Esp	Emparejamiento

It	Accoppiare
NL	Paarvorming
עברית	התאמה
Svenska	Parning


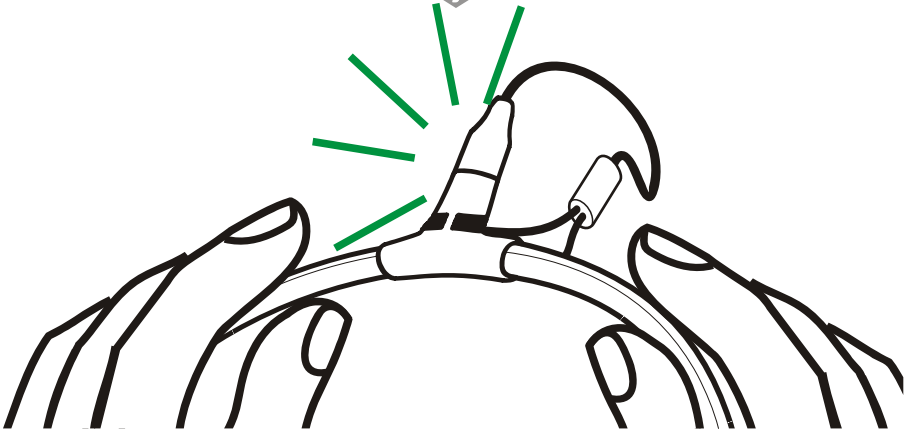
j


k

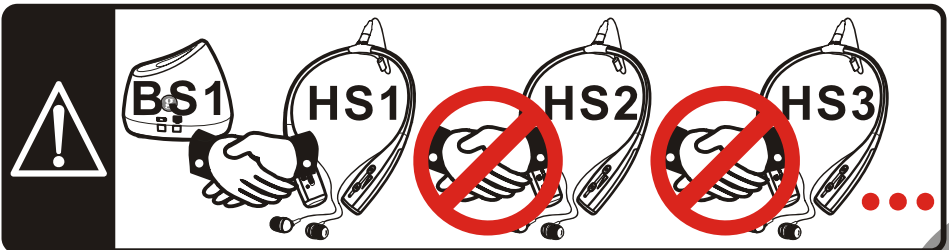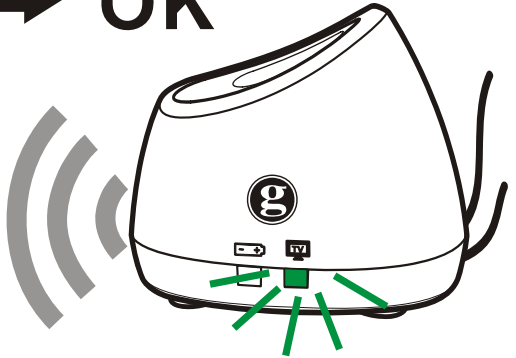
ON → OK

En	Volume	It	Volume
Fr	Volume	NL	Volume
De	Lautstärke	עברית	עוצמת שמע
Esp	Volumen	Svenska	Volym

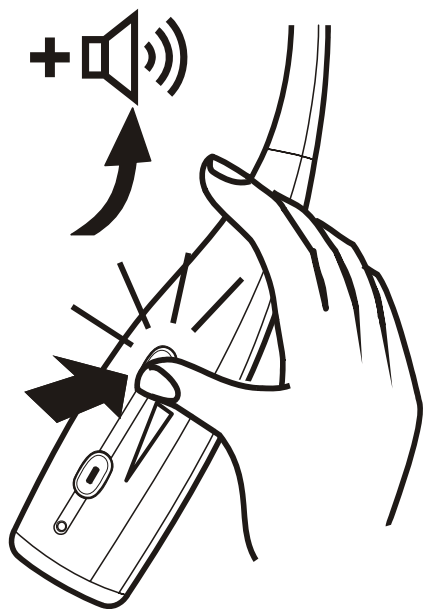Max

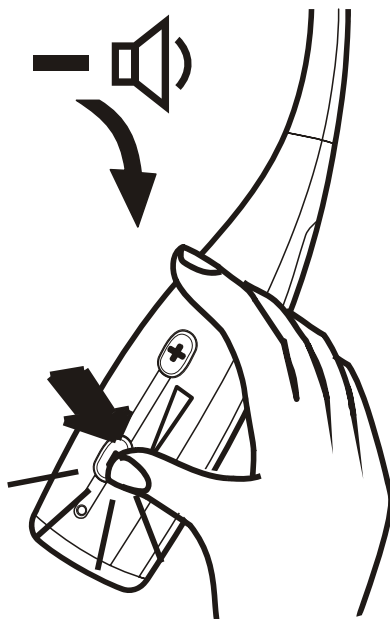Min

En	How to use
Fr	Utilisation
De	Bedienungsanleitung
Esp	Cómo se usa

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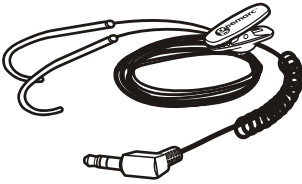
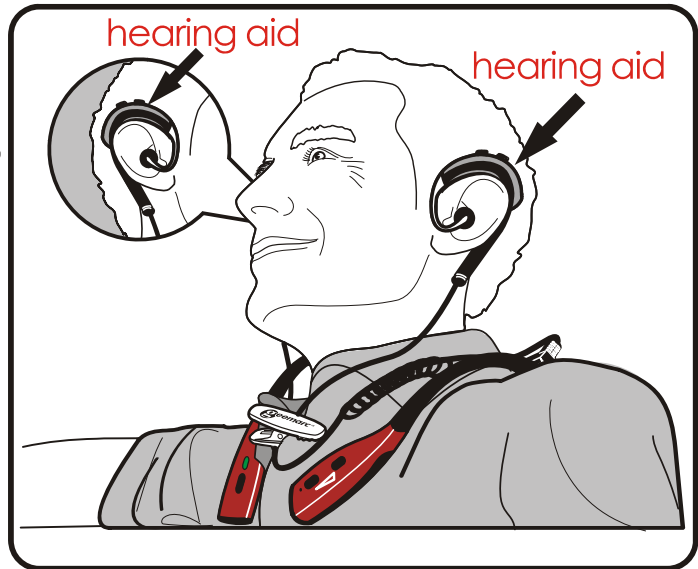
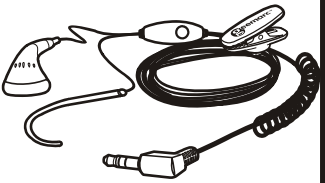
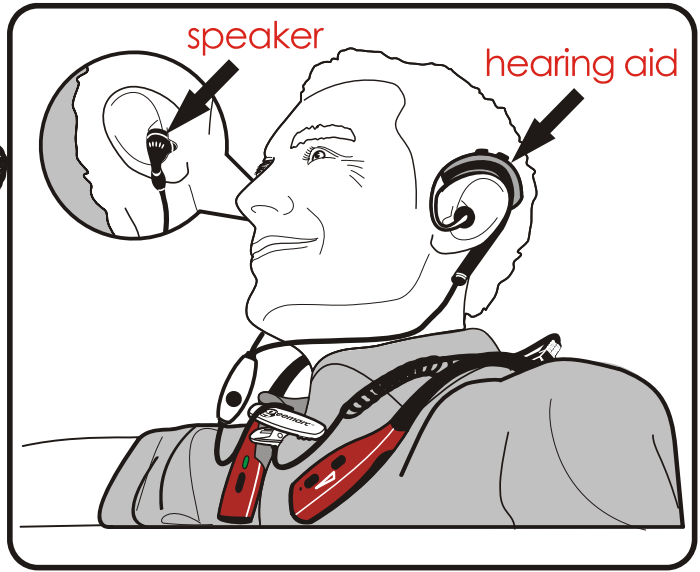
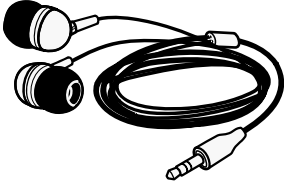
Extra headset - To be defined

Safety & Care advice

To make the best use of this product and to use it safely, please read these instructions thoroughly before use and keep safe for future reference.

Warning: This product can be very loud. Take care to prevent others from using this product, if they do not have the same level of hearing loss. Excessive sound pressure from earphones and headphones can cause hearing loss.

Caution: Electric shock. DO NOT insert unattached plug heads in to mains power sockets. Plug heads **MUST** be fitted to the power adaptor.

Electrical connection: The unit is designed to operate from a 100-240VAC - 50/60Hz supply. (Classified as 'hazardous voltage' according to EN 60950 standard).

The unit does not incorporate an integral power on/off switch. To disconnect the power, either switch off supply at the mains power socket or unplug the AC adaptor. When installing the unit, ensure that the mains power socket is readily accessible.

Caution: Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.

Keep the unit away from heat sources.

DO NOT expose the unit to direct sunlight.

Clean the Headset and ear pads regularly.

DO NOT modify the unit in any way.

Please note: The unit may be subject to interference from appliances that generate strong electrical or magnetic fields e.g. microwaves, mobile phones, lighting transformers etc.

Battery Warnings:

- **DO NOT** attempt to dismantle.
- **DO NOT** dispose of in fire.
- **DO NOT** swallow.
- Keep away from children.
- Batteries are to be inserted with the correct polarity.
- **DO NOT** short-circuit the supply terminals.
- Exhausted batteries are to be removed from the product.
- Only use Geemarc supplied **2/3AAA** batteries.
- **Caution:** Risk of explosion if battery is replaced incorrectly or fitted with the wrong type of battery. Dispose of used batteries according to the instructions.

Customer support

For product support and help, visit our website at www.geemarc.com

For our Customer Helpline

Telephone: **01707 384438**

Fax: 01707 832529

General information

Specifications

- Digital clear sound with 44.1 KSPS, 16 bit & 4Mbps RF link
- Reception adjustment up to 48dB
- 2.4 GHZ ISM band solution
- Frequency response : 40 - 15000Hz
- Up to 2 hour charge. Headset recharges itself when paced back on top of transmitter
- Operates up to 15 meter indoor from the transmitter
- working continuous time : more then 5 hours
- 32Ω(ohms) earphones
- 3.5 mm audio input stereo jack in transmitter with different adaptors to RCA or SCART connectors

Guarantee

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our Helpline or visit our website at www.geemarc.com.

The guarantee does not cover accidents, negligence or breakage to any parts.

The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

The product is covered by the legal guarantee of conformity as provided by applicable law.

Important: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

EEC Declaration

Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5 EEC.

The declaration of conformity may be consulted at www.geemarc.com

Disposal information

Disposal of old electrical & electronic equipment (applicable in the European Union and other European countries with separate collection systems)

The symbol on the product or its packaging indicates that this product may not be treated as household waste. Instead it should be handed over to the applicable collection point for the recycling of electrical and electronic equipment.

By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

For more detailed information about recycling this product, please contact your local council office, your household waste disposal service or the shop where the product was purchased.

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