geemarc™ AMPLICALL™ 150
Hear the Difference

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Français P8

Deutsch P15

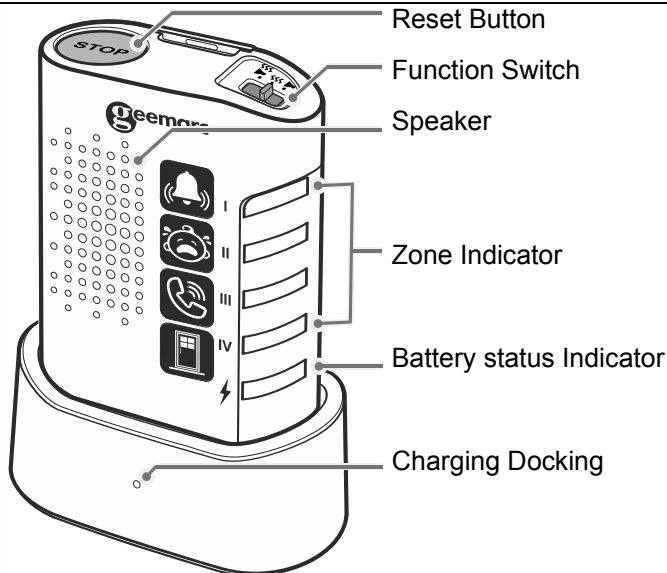
Please refer to our website: www.geemarc.com for an up to date user guide, as there may be important updates and changes you need to be aware of.

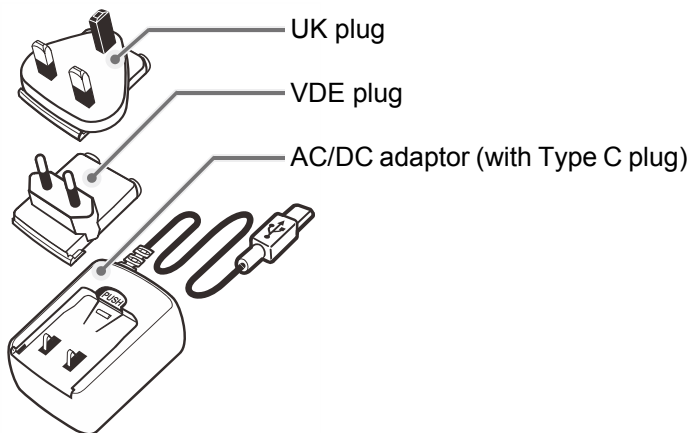
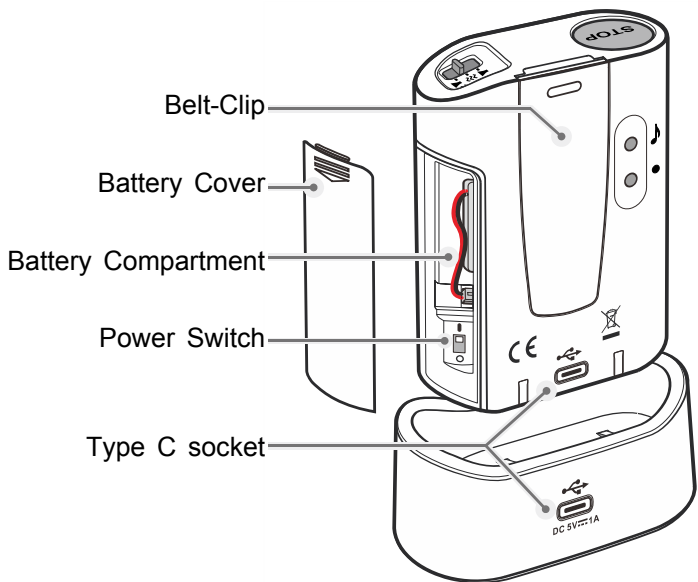
PACKAGE CONTENTS

The AMPLICALL™150 box contains the following items:

- ✧ 1 Base station AC150 (AMPLICALL™150)
- ✧ 1 charging cradle
- ✧ 1 rechargeable battery (pre-installed)
- ✧ 1 AC/DC power adaptor
- ✧ 1 UK and 1 VDE plug adaptors
- ✧ 1 user's manual
- ✧ 1 set (8 pcs) alarm icon stickers

DESCRIPTION


SWITCH ON/OFF

The receiver built-in a replace lithium polymer battery. When using for the first time, you need to turn on the power switch inside the battery cover **ON** (I).

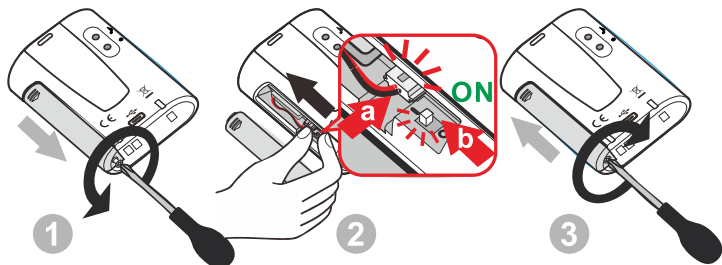When not used for a long period, it is recommended to turn the switch to the **OFF** (O) position.


Loosen the screw with a small phillips screwdriver.

Connect the battery and slide the switch to **on** position.


Close the battery cover and tight the screw with the screwdriver.

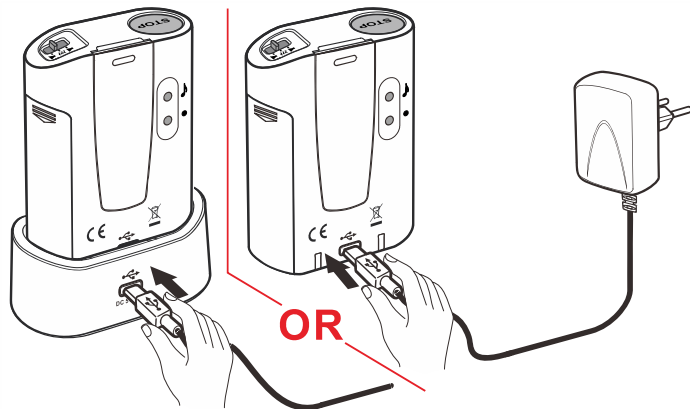BATTERY CHARGING

Charging with Docking – Connect the type C USB plug to the docking, put the AC150 receiver to the docking.

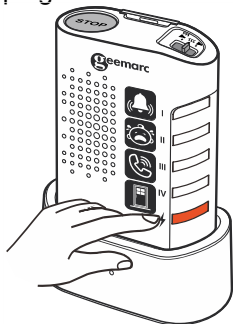Charging without Docking – Direct connect the type C USB plug to the AC150 receiver unit.

Battery status indicator


	Charging Mode	Standby Mode
LED Light	Charging	
LED OFF	Full	Normal
LED Blink each 10S		Local Battery Low

Battery low indicator – If a zone LED light is blinking every 10 S, it indicates the battery is low for this transmitter. You must recharge the battery as soon as possible.

PAIRING & ALARM MELODY

To pair a new device, please follow these steps:

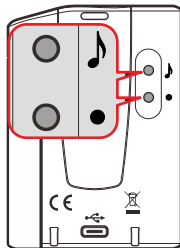1. Enter pairing mode for the AC150.
Press & hold the **LEARNING** button (●) for 3 seconds. The LED in zone #1 will flash.
2. Press the **LEARNING** button (●) again to move to the zone you wish to pair with.
Press the **melody selection** (🎵) button to scroll through the melodies. Once you hear a melody you like, do not press any button during the melody.
3. Once the zone and melody are selected, press one time on device (AC101, AC110, AC140) button to pair. If successful, the AMPLICALL150 will generate two short tones.
4. Press the **LEARNING** button (●) to exit the pairing mode; the system is now ready to use.

Remove and transmitter of the AC150 memory

- ✧ First, press and hold the (●) button for 3 seconds to enter the pairing mode.
- ✧ Press the (●) button to move to the zone which is to be deleted.
- ✧ Press and hold the (●) and (🎵) buttons together for 3 seconds
The receiver will generate a tone to indicate the transmitter of this zone has been deleted.

CHANGE THE MELODY IN DESIRED ZONE

- ✧ First, press and hold the (●) button for 3 seconds to enter the pairing mode.
- ✧ Press the (●) button to move to the zone then press the (🎵) button to select the melody required.
- ✧ Press the (●) button until the pairing mode is exited.

USING

When the AC150 receives a trigger signal, it will play a melody and vibrating (depend on the Alarm setting position). The zone LED will light to show which sensor is triggered. The Zone LED will keep lighting until the reset button is pressed..

GUARANTEE

From the moment your Geemarc AMPLICALL™150 is purchased, Geemarc guarantee is for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at www.geemarc.com. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

DECLARATIONS:

UKCA Directives

Geemarc Telecom SA hereby declares that this AMPLICALL™150 is in compliance with the essential requirements and other relevant provisions of the Radio Equipment Directive 2014/53/UE.

The UKCA declaration of conformity may be consulted at www.geemarc.com

RF power : < 250mW

Recycling Directives

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it. Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website at www.geemarc.com

E-mail : help@geemarc.com

telephone **01707 387602**

lines are open 09h00 to 16h00 Mon to Fri

2 Rue Galilée, Parc de l'Etoile, 59791 GRANDE-SYNTHE Cedex,
FRANCE

This product is made in China To Geemarc Telecom

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