



QUICK USER GUIDE

En

!!! Please refer to our website: www.geemarc.com for an up to date user guide, as there may be important updates and changes you need to be aware of !!!

CAUTION: Use only the battery provided. There may be a risk of explosion if a wrong type of battery is used for the parent unit. The battery cannot be subjected to high or low extreme temperature and low air pressure at high altitude during use, storage or transportation. Disposal of battery into fire or a hot oven, or mechanically crushing or cutting or the battery can result in an explosion. Leaving the battery in an extremely high temperature surrounding environment can result in an explosion or the leakage of flammable liquid of gas. Battery subjected to extremely low air pressure may result in an explosion or the leakage of flammable liquid or gas. Dispose of used battery according to the instructions.

Normal operating temperature for the product (e.g. 0 °C ~ 40°C).

Use only the adapters and battery listed below.


Power adapter information: output: 6V DC 0.45A, 2.7W. Vtech Telecommunications Limited. Model: VT05EEU06045 (EU), VT05EUK06045 (UK)

Rechargeable Battery: Model: KH-AAAJ600-D x 3pcs. 1.2V DC, 600mAh. Corun.

PACKAGE CONTENTS


1 Charger, 1 cordless Handset, 1 power adapter, 3 NiMh batteries, 1 Quick user guide.

CONNECTION BASE

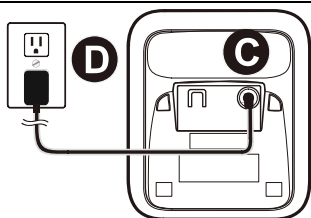

1

HANDSET

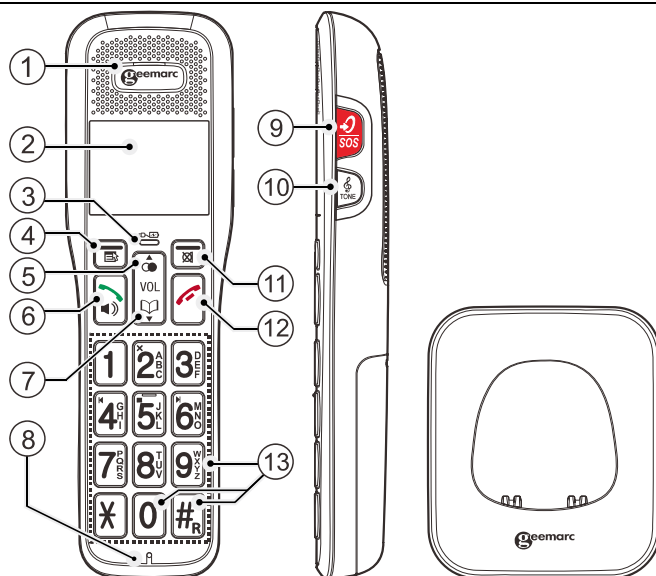

INSTALLATION

Connecting the telephone

Connect the power adapter jack to the base (C) then connect the power adapter plug in mains power (D).

Note: Use only the power adapters supplied with the phone.

Connecting the battery

Slide the battery cover downwards.

Install the 3 batteries into the socket as indicated, matching correct polarity (+,-).

Align the cover flat against the battery compartment, and then slide it upwards until it clicks into place.

Charging the batteries

Charge the batteries for at least 16 hours when charging for the first time.

FIRST USE OF YOUR HANDSET

After you have inserted the batteries, the display asks you to select your text language with the keys ▼/▲(5/7), Press MENU/OK (4) to valid.

Then display REGISTER, make the procedure at the chapter "HANDSET REGISTRATION".

Then enter the date with the keypad and validate by pressing MENU/OK (4).

Enter the correct hour with the keypad and validate by pressing MENU/OK (4).

Select your talking language with the keys ▼/▲(5/7), Press MENU/OK (4) to valid.

HANDSET REGISTRATION

Note: Up to four handsets can be registered to a single base station.

Registering a handset

On main **Base:** press and hold /HANDSET LOCATOR on the base until you hear a beep to enter the registration mode.

At the same time **On Handset** it is display REGISTER: Press MENU/OK (4) .

REGISTERING will be displayed.

After a short delay, if the registration is successful, there will be a confirmation beep, and the handset display will go back to idle mode with the new handset number shown. The base display will also go back to idle mode.

OVERVIEW

Handset

1. EARPIECE

2. LCD DISPLAY

3. CHARGE LIGHT ()

Steadily on when the handset is charging in the charger.

2

4. MENU/OK ()

In idle mode: Press to access the main menu.

In sub-menu mode: Press to confirm the selection.

During a call: Press to access Intercom / Phonebook / Redial list / Call log.

5. UP/CALL LIST ()

In idle mode: Press to access the redial list.

In menu mode: Press to scroll up the menu items.

In phonebook list/redial list/call list: Press to scroll up the list.

In editing mode: Press to move the cursor one character to the left.

During a call: Press to increase the listening volume.

During ringing: Press to increase the ringer volume.

6. TALK ON () SPEAKERPHONE ()

In idle/pre-dial mode: Press to make a call in earpiece mode.

In phonebook/redial list /call list: Press to make a call to the selected entry in the list in earpiece mode.

During ringing: Press to answer a call in earpiece mode.

On line: Press again to pass in speakerphone mode. Press again to switch to earpiece mode.

7. DOWN/PHONEBOOK ()

In idle mode: Press to access the phonebook.

In menu mode: Press to scroll down the menu items.

In phonebook list/redial list/call list: Press to scroll down the list.

In editing mode: Press to move the cursor one character to the right.

During a call: Press to decrease the listening volume.

During ringing: Press to decrease the ringer volume.

8. MICROPHONE

9. VOLUME BOOST/SOS (/SOS)

During a call: Press to activate/deactivate the VOLUME BOOST function.

In idle mode: Press and hold for 3 seconds to enter the emergency calling sequence set in your handset.

In call mode: press and hold for 2 seconds to stop the emergency call.

10. TONE ()

During a call: press this key to increase the high frequency or the low frequencies.

11. CANCEL/MUTE ()

In main menu mode: Press to go back to idle screen.

In sub-menu mode: Press to go back to previous menu.

In editing/pre-dial mode: Press to delete a character/digit.

In editing/pre-dial mode: Press and hold to delete all the characters / digits.

During a call: Press to mute/resume the microphone.

During ringing: Press to silence the handset ringer.

In idle mode: Press to intercom another handset.

12. TALK OFF ()

During a call: Press to end a call and go back to idle screen.

In menu/editing mode: Press to go back to idle screen.

13. ALPHANUMERIC KEYPAD, STAR (0), HASH (#R)

In predial or editing mode: Press to insert a digit / character /~~X~~/#.

#R during a call: Press and hold to make a Flash (Recall).

0 key in idle/pre-dial mode/number editing mode: Press and hold to insert a pause.


FUNCTIONS

You can find all the functionalities in the main user guide AMPLIDECT595 U.L.E.

INTERCOM

Internal Call

1) Press **MENU/OK** (), and **▼▲** to select **INTERCOM**, Press **MENU/OK** (),

2) Choose the desired handset number and press **MENU/OK** (). If you receive a call on the base in speaker mode, the call is automatically transferred to the handset if the handset is removed


from the charger.

Call All Handsets

Make point 1 in chapter internal call and choose **ALL**.


Transfer an External Call

Make point 1 in chapter internal call and choose the desired handset number with the key **UP/DOWN**.

After connection quit the call on handset by pressing .

Conference Call

Make point 1 in chapter internal call and choose the desired handset number with the keys **▼/▲**.

After connection press and hold  until CONFERENCE is shown.

NOTE: Any party that hangs up during a conference call will leave the other party still connected with the external call.

See the user guide of the additional handset.

MAINTENANCE

Clean the surface of the housing with a soft, fluff-free cloth. Do not use any cleaning agents or solvents.

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee is for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at www.geemarc.com.

The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.


DECLARATIONS:

UKCA Directives: Geemarc Telecom SA hereby declares that this AMPLIDECT595 U.L.E Handset is in compliance with the essential requirements and other relevant provisions of the Radio Equipment Directive 2014/53/UE.

The declaration of conformity may be consulted at www.geemarc.com

RF power	< 250mW
----------	---------

Power Supply

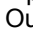Charger	Input 100-240V 50/60Hz; Output DC6V  0.45A, 2.7W adapter
---------	---

Handset	3 batteries AAA 1.2V/600mAh rechargeable
---------	--

Recycling Directives

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip.

Deposit the product in an appropriate WEEE skip.

- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

If you respect these instructions you ensure human health and environmental protection.



For product support and help visit our website at www.geemarc.com

E-mail : help@geemarc.com

telephone **01707 387602**

lines are open 09h00 to 16h00 Mon to Fri

Parc de l'étoile, 2 rue Gaillée, 59760 Grande-Synthe, France

Made for Geemarc Telecom S.A. in China



QUGAMPLIDECT595 U.L.E_HS_En_A4_V1.5